

Custom options — technical reference guide


As a value added service to our customers, Solventum MedTech OEM provides custom converting on our tape products to meet customer needs. One service is converting the tape into widths and lengths specific to our customer's requirements. We also can provide slit liner (SL) and/or extended liner (EL) on many of our tape products.

Slit liner and extended liner capabilities


Slit liners (SL) — liner is “scored” (not cut through totally) or “slit” through to (but not through) the tape

- Available on all products on a release liner except:
 - Transfer adhesives #1504, #1524 and #1524A
 - Polyurethane tapes & film #9832F, #9833, #9834 and #9836.
- Tolerance is $\pm 1/16"$ ($\pm 1.5\text{mm}$)
- Standard maximum slit roll width is 24" (61cm) and minimum slit width is 0.5" (1.27cm)
- Slit (or score) must be a minimum of 1/8" (3mm) from the roll edge
- Orders must include total number and location(s) of slit(s) and distance from the left edge

Single-coated tapes

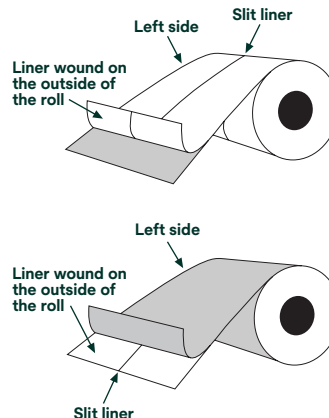


Double-coated tapes

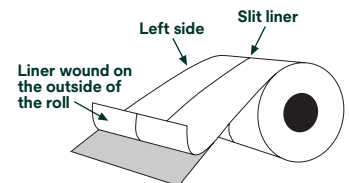


Extended liners (EL) — liner extends past the edge or edges of the adhesive

- Available on all products with a release liner except:
 - any foam tape
 - Transfer adhesives #1524 and #1524A
 - Hydrogel adhesive #9880
- All other products have an EL minimum width of 1/8" (3mm) and tolerance is $\pm 1/16"$ ($\pm 1.5\text{mm}$)
- Standard maximum roll width is 24" (61cm) and minimum slit width is 0.5" (1.27cm)

Single-coated tapes

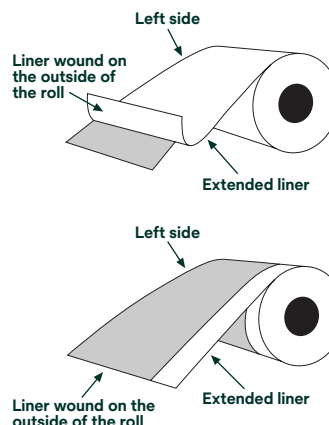

Double-coated tapes

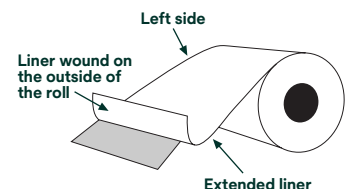

Note: Please be aware that there may be some restrictions on maximum roll length of any product slit less than 1" (25.4mm) wide due to roll stability during handling.

Product and safety information: User is solely responsible for determining the suitability of Solventum samples and products for the intended use including any necessary safety or toxicity assessment. Solventum will provide Material Safety Data Sheets or equivalent and summary results of biocompatibility testing upon request. In every case before using any product in full scale production users should conduct their own tests to determine to their own satisfaction whether the product is of acceptable quality and is suitable for their particular purposes under their own operating conditions.

Notice: Nothing contained herein shall be construed to imply the nonexistence of any relevant patents or to constitute a permission, inducement or recommendations to practice any invention covered by any patent, without authority from the owners of this patent.

Warranty information

All statements, technical information and recommendations herein are based on tests Solventum believes to be reliable, but the accuracy or completeness thereof is not guaranteed. Solventum warrants only that products will meet Solventum's specifications at the time of shipment to the customer. Solventum does not offer any other warranty and does not warrant the performance, safety or such other characteristics of products in combination with other materials. Solventum specifically **does not** warrant products for any intended or unintended uses (whether or not foreseeable); for compatibility or suitability with other components or compatibility with any methods of manufacture or conversion. The foregoing warranty is made in lieu of all other warranties, expressed or implied, including the implied warranties of merchantability, fitness for a particular purpose and freedom from non-infringement.

Limitation of remedies: If products are proven not to meet Solventum's specifications, the sole and exclusive remedy available and Solventum's only obligation shall be, at Solventum's option, to replace such quantity of products which are proven out of specification or to refund the purchase price paid for products.

Limitations of liabilities: The remedies provided herein are exclusive remedies against Solventum for any alleged or actual nonconformance to specifications or defect or other failure in products or for Solventum's performance of its supply obligations. Under no circumstances is Solventum liable for any direct, indirect, incidental, special or consequential damages (including lost profits) in any way related to the product under any theory of law including, but not limited to, negligence and strict liability.

Ordering information

Call us at Customer Service to place an order: 800-742-1994 (U.S.).

Visit our website: go.Solventum.com/MedTechOEM for product and services information, news about conferences we will be attending, new product highlights or to make a direct inquiry.

To have a sales representative contact you, to request samples or clinical and safety summaries, please contact us at Solventum Helpline 800-742-1994 (U.S.) or for international inquiries, please contact your local country representative.



Solventum MedTech OEM

Phone 1-800-742-1994

Web go.Solventum.com/MedTechOEM

© Solventum 2025. Solventum, the S logo and other trademarks are trademarks of Solventum or its affiliates.

116390