

How to use


Disinfecting Cap for Home Infusion

Disinfecting Cap for Curos™ Needleless Connectors

Disinfects in 1 minute. Protects ports for up to 7 days



Peel the foil from the Curos Disinfecting Cap

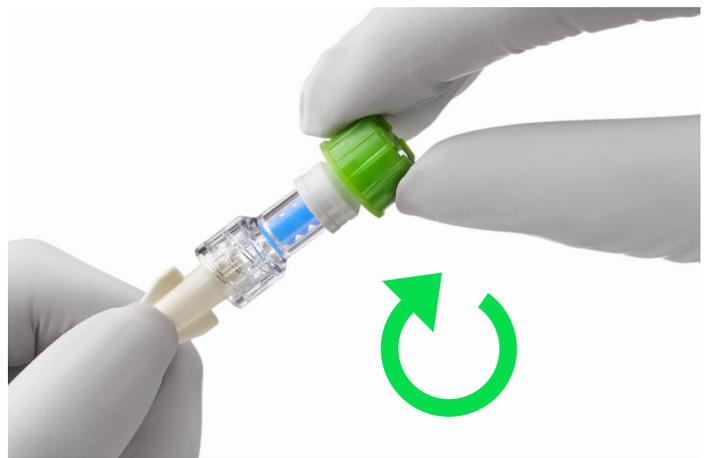

Push and twist the Curos disinfecting cap clockwise onto the end of the needleless connector

- If the valve becomes soiled with blood or other contaminants, it should be swabbed with a 70% isopropyl alcohol pad for 15 seconds before placing a new Curos cap on the clean valve.
- As a precaution, replace the Curos cap if it gets wet.
- Do not use cap if seal is broken, torn, punctured, or sponge appears dry.
- Cap must remain on needleless connector for a minimum of one (1) minute and may remain on for seven (7) days if not removed.
- Always place a new Curos disinfecting cap on needleless connector following completion of a valve access.
- Dispose of the Curos disinfecting cap after every use.

Warning: To avoid potential for injury, use only on needleless connectors

Caution: Potential choking hazard



Green means clean

STERILE R

Active Ingredient: 70% isopropyl alcohol (IPA). Single Use Only. Rx Only. This device is not made with natural rubber latex or plasticiser Diethylhexylphthalate (DEHP). Refer to current device labeling for indications for use, warnings and product details.

© Solventum 2025. Solventum and the S logo are trademarks of Solventum or its affiliates. 3M and the 3M logo are trademarks of 3M. All other trademarks are owned by their respective owners.

Solventum Medical Surgical

Phone 800-228-3957
Web Go.Solventum.com/Curos

70-2011-5736-2