



The science of fiberglass. The art of healing.

Water-activated polyurethane resin



Solventum Casting and Splinting Products

Mastering the art of molding wrinkle-free casts and splints takes practice. Working with the fiberglass technology in Solventum casting tapes and splints can make it easier.



Reaches full weight-bearing strength in just 20 minutes



Easy to mold, with plaster-like conformity



Lightweight and durable for patient comfort



3M™ Scotchcast™ Plus Casting Tape

- Water-activated resin cures in 20 minutes
- Rigid, durable cast is lightweight for easy mobility
- Conforms to contours for a better fit and comfort
- Open weave allows skin to breathe

Casting Tapes

When you need a durable or flexible cast, the moldable characteristics of 3M™ Scotchcast™ Casting Tapes make changing direction easy for casting around the patient's bony prominences.



3M™ Scotchcast™ Soft Cast Casting Tape

- Specialty product ideally suited for Total Contact Casts (TCC), club foot, and sports injuries
- Water-activated resin remains flexible when cured
- Lightweight
- Soft, flexible cast allows for micro motion
- Soft, pliable cast edges improve patient comfort
- Can be used in combination with rigid fiberglass splints

Use 3M™ Scotchcast™ Casting Tapes with:



3M™ Synthetic Cast Stockinet



3M™ Scotchcast™ Wet or Dry Cast Padding

or



3M™ Synthetic Cast Padding

Casting tape ordering information

Cat. no.	Description	Width	Length	Rolls/case
3M™ Scotchcast™ Plus Casting Tape				
82001	White	1" (2,5 cm)	2 yds. (1,8 m)	10
8200*	White	2", 3", 4", 5" (5 cm, 7,5 cm, 10 cm, 12,5 cm)	4 yds. (3,6 m)	10
8200*A	Black	2", 3", 4" (5 cm, 7,5 cm, 10 cm)	4 yds. (3,6 m)	10
8200*B	Blue	2", 3", 4", 5" (5 cm, 7,5 cm, 10 cm, 12,5 cm)	4 yds. (3,6 m)	10
8200*G	Green	2", 3", 4" (5 cm, 7,5 cm, 10 cm)	4 yds. (3,6 m)	10
8200*L	Light Blue	2", 3", 4" (5 cm, 7,5 cm, 10 cm)	4 yds. (3,6 m)	10
8200*R	Red	2", 3", 4" (5 cm, 7,5 cm, 10 cm)	4 yds. (3,6 m)	10
8200*U	Purple	2", 3", 4" (5 cm, 7,5 cm, 10 cm)	4 yds. (3,6 m)	10
8200*V	Bright Green	2", 3" (5 cm, 7,5 cm)	4 yds. (3,6 m)	10
8200*W	Bright Orange	2", 3" (5 cm, 7,5 cm)	4 yds. (3,6 m)	10
8200*X	Bright Pink	2", 3", 4" (5 cm, 7,5 cm, 10 cm)	4 yds. (3,6 m)	10
8200*Y	Yellow	2", 3" (5 cm, 7,5 cm)	4 yds. (3,6 m)	10
3M™ Scotchcast™ Soft Cast Casting Tape				
82101	White	1" (2,5 cm)	2 yds. (1,8 m)	10
8210*	White	2", 3", 4", 5" (5 cm, 7,5 cm, 10 cm, 12,5 cm)	4 yds. (3,6 m)	10
8210*B	Blue	2", 3" (5 cm, 7,5 cm)	4 yds. (3,6 m)	10
8210*U	Purple	2", 3", 4" (5 cm, 7,5 cm, 10 cm)	4 yds. (3,6 m)	10
8210*R	Red	2", 3", 4" (5 cm, 7,5 cm, 10 cm)	4 yds. (3,6 m)	10

Important: When ordering 3M™ Scotchcast™ products, please replace the asterisk () with the selected roll width. For example, 8200*L would become 82003L for 3" Light Blue material.

3M™ Scotchcast™ Plus Casting Tape color options



3M™ Scotchcast™ Soft Cast Casting Tape color options



Roll splints and pre-cut splints

Splinting is the first intervention following injury. The seven layers of fiberglass in 3M™ Scotchcast™ Splints conform easily around the bends and curves of the patient’s body. The fiberglass splints set in 3–4 minutes and cure in 20 minutes to achieve full strength. The water-repellent technology in 3M splint padding sheds water and is comfortably dry within 1–3 hours, reducing the risk of skin maceration.*

*3M data on file



3M™ Scotchcast™ Quick Step Splint

- Made with thin, water-repellent felt padding to reduce moisture against the patient’s skin
- Felt covering on both sides making it easier to apply and impossible to put on incorrectly
- High-density felt prevents resin bleed through

Splint ordering information

Cat. no.	Size	Packaging
3M™ Scotchcast™ Quick Step Double Sided Felt Roll Splint		
74002Q	2" x 15' roll	1 roll/box, 2 box/case
74003Q	3" x 15' roll	1 roll/box, 2 box/case
74004Q	4" x 15' roll	1 roll/box, 2 box/case
74005Q	5" x 15' roll	1 roll/box, 2 box/case
3M™ Scotchcast™ Quick Step Double Sided Felt Roll Splint		
74002Q	2" x 15' roll	1 roll/box, 2 box/case
74003Q	3" x 15' roll	1 roll/box, 2 box/case
74004Q	4" x 15' roll	1 roll/box, 2 box/case
74005Q	5" x 15' roll	1 roll/box, 2 box/case
3M™ Scotchcast™ Quick Step Precut Splint Splint		
75210	2" x 10" (5,1 cm x 25,4 cm)	10 each/case
75312	3" x 12" (7,6 cm x 30,5 cm)	10 each/case
75335	3" x 35" (7,6 cm x 88,9 cm)	10 each/case
75415	4" x 15" (10,1 cm x 38,1 cm)	10 each/case
75430	4" x 30" (10,1 cm x 76,2 cm)	10 each/case

Accessories

3M also offers a complete line of casting accessories, including wet or dry cast padding, synthetic cast padding, and stockinet—all to help you work efficiently and achieve reliable outcomes.



3M™ Scotchcast™ Wet or Dry Cast Padding

- If it gets wet, can dry within 1–3 hours*
- Porous, synthetic material always stays soft, never gets stiff
- Keeps its padding ability
- Water-repellent coating



3M™ Synthetic Cast Stockinet

- Underlayment for all standard casting and splinting applications
- Suitable for use with synthetic and plaster casting materials
- An essential component of a high-quality cast



3M™ Synthetic Cast Padding

- Suitable for use with synthetic and plaster casting materials
- Porous, synthetic material always stays soft, never gets stiff
- Keeps its padding ability

Accessories ordering information

Cat. no.	Width	Each/bag	Bag/case
3M™ Scotchcast™ Wet or Dry Cast Padding			
WDP2	2" x 4 yds. (5 cm x 3,6 m)	20	4
WDP3	3" x 4 yds. (7,5 cm x 3,6 m)	20	4
WDP4	4" x 4 yds. (10 cm x 3,6 m)	20	4
WDP6	6" x 4 yds. (15 cm x 3,6 m)	20	4
3M™ Synthetic Cast Padding			
CMW02	2" x 4 yds. (5 cm x 3,6 m)	20	4
CMW03	3" x 4 yds. (7,5 cm x 3,6 m)	20	4
CMW04	4" x 4 yds. (10 cm x 3,6 m)	20	4
CMW06	6" x 4 yds. (15 cm x 3,6 m)	20	4
3M™ Synthetic Cast Stockinet			
MS01	1" x 25 yds. (2,5 cm x 22,8 m)		1
MS02	2" x 25 yds. (5 cm x 22,8 m)		1
MS03	3" x 25 yds. (7,5 cm x 22,8 m)		1
MS04	4" x 25 yds. (10,1 cm x 22,8 m)		1
MS06	6" x 25 yds. (15,2 cm x 22,8 m)		1

For product questions or to request a sample, call 1-800-228-3957 or visit [3M.com/immobilization](https://www.3m.com/immobilization). To place an order, contact your distributor sales representative.



Solventum
2510 Conway Ave E
Maplewood, MN 55144
USA

Phone 855-423-6725
Web [Solventum.com](https://www.Solventum.com)

Solventum
Germany GmbH
Edisonstraße 6
59174 Kamen
Germany

© Solventum 2025. Solventum, the S logo and Scotchcast, are trademarks of Solventum or its affiliates. 3M and the 3M logo are trademarks of 3M. Other trademarks are the property of their respective owners.

70-9700-0049-6 Rev 08/2025