



3M™ Attest™ Vaporized Hydrogen Peroxide Tri-Metric Chemical Indicator, Type 4


Unprocessed

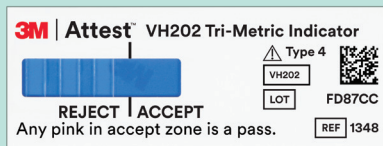

Processed

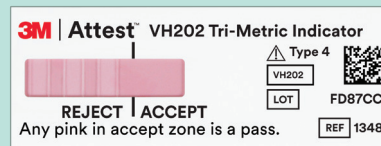

Accept

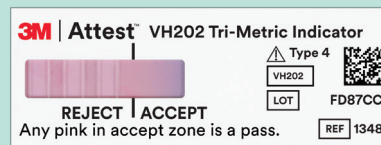

Accept

A colour change toward pink has reached the **ACCEPT** zone in the two samples shown above, verifying that the stated values for the three critical parameters of exposure time, temperature and concentration of vaporized hydrogen peroxide have been achieved.

If a colour change toward pink has entered the ACCEPT zone, it is considered a pass.

Processed



Reject



Reject

A colour change toward pink has not entered the **ACCEPT** zone in the two samples shown above, a **REJECT** result is indicated, the items in the pack, container system or tray were not exposed to sufficient vaporized hydrogen peroxide sterilization conditions.

The pack should be reprocessed and the cause of the failure should be investigated.

If a colour change toward pink has not entered the ACCEPT zone, it is considered a fail.