




3M™ Tegaderm™ I.V. Advanced Securement Dressing 1683

Comprehensively designed to help you deliver exceptional patient care through advanced catheter securement, enhanced wear time and gentle removal. This dressing is an option for smaller vascular access sites in pediatrics and adults and is safe to wear up to 7 days.

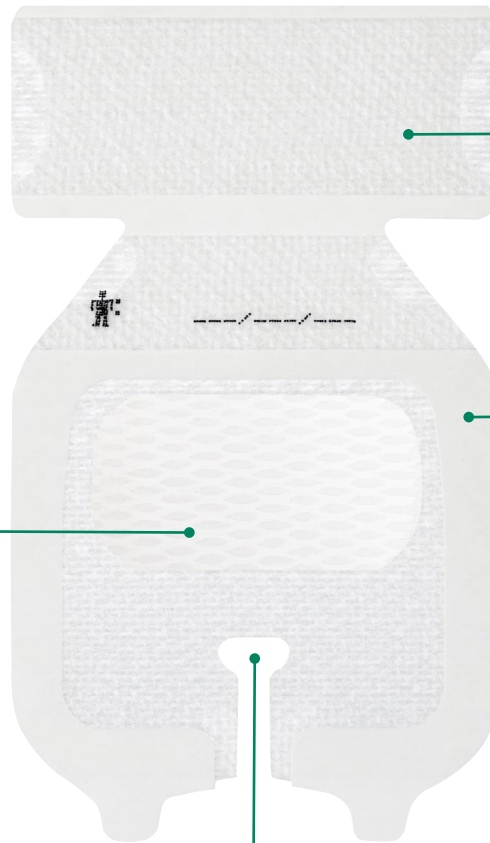

A waterproof, sterile barrier protects against external contaminants*
 Highly breathable, transparent film.

Large securement tape strip
 Promotes consistent application and enhances stabilization.

Conforming edge border
 Uses technology designed to reduce edge-lift.

Conforming keyhole notch
 A notch allows catheter lumens to fit better and stay in place.

Site visibility

Transparent dressing enables early identification of complications at the insertion site.

Catheter securement

Designed to minimize catheter movement and dislodgement.

Ease of use


Frame delivery makes placement accurate and easy for consistent application. Enables one handed application.

Patient comfort


Safe to wear up to 7 days, potentially reducing the number of dressing changes and restarts.

*In vitro testing shows that the film provides a barrier against viruses 27 nm in diameter or larger while the dressing remains intact without leakage.

1683

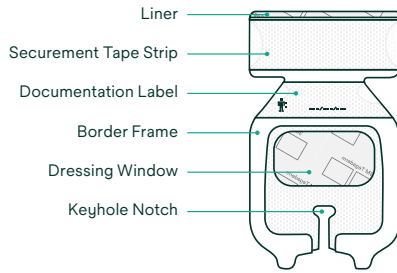


Prepare

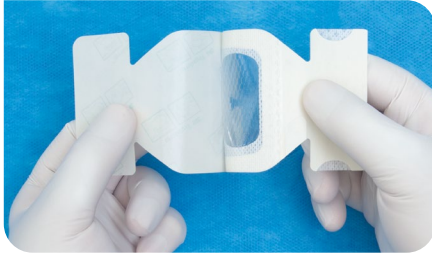


Prepare site according to facility protocol. Ensure site is completely dry.

Press

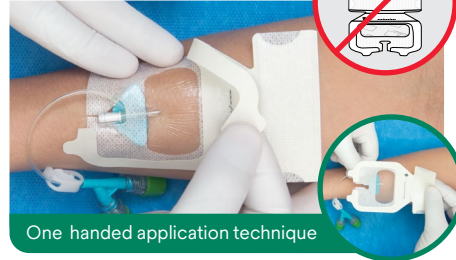

Do not stretch the dressing at placement. Peel the liner from the dressing and place the dressing window over insertion site, aligning extension tubing with securement notch. Apply firm pressure to entire dressing with one hand for optimal adhesion, while removing border frame with opposite hand.

Secure

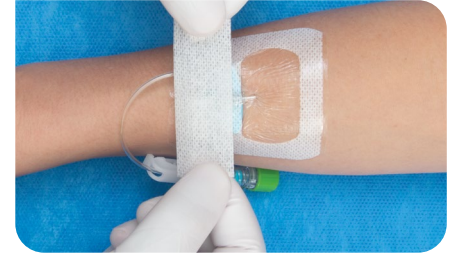

Align the securement tape strip with the lower edge of the dressing. Apply securement tape strip over the extension tubing.

Secure



Document the dressing change information on the label strip. Tape strip may be used to secure the extension tubing.

Monitor



Change the dressing:

- At least every 7 days
- If the dressing becomes loose, soiled or damp
- According to facility protocol

Removal



Remove tape strips. Using a low and slow removal technique, start removing the dressing from where the catheter or tubing exits the dressing toward the catheter insertion site. Avoid skin trauma by peeling the dressing back rather than pulling it up from the skin.

Refer to product Instructions for Use for other important information.

Ordering information

| Product name | Product number | Overall dressing size | Dressings/box | Boxes/case |
|--|----------------|-------------------------------------|---------------|------------|
| 3M™ Tegaderm™ I.V. Advanced Securement Dressing 1683 | 1683 | 2 1/2 in x 2 3/4 in (6,5 cm x 7 cm) | 100 | 4 |



Visit Go.Solventum.com/IVSMTraining to find product training resources for all Solventum IV care solutions.



Solventum Medical Surgical

Phone 800-228-3957
Web Go.Solventum.com/Medical

© Solventum 2026. Solventum and the S logo are trademarks of Solventum or its affiliates. 3M is a trademark of 3M.

70-2011-8088-5