

3M™ RelyX™ Universal Resin Cement

Pre-treatment and priming recommendations by material class and indication¹


3M™ Scotchbond™
Universal Plus
Adhesive

- Rub in 20 sec
- Gently air dry 5 sec



- 1 Pre-treat restoration
- 2 Prime restoration
- 3 Etch tooth
Phosphoric acid
- 4 Prime tooth
- 5 Apply cement and seat

Zirconia, metal

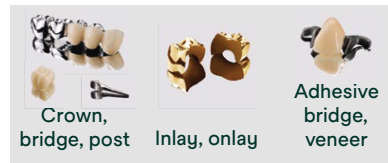

Sandblast
Alumina 30 – 50 µm, 2 bar (30 PSI)

		•
	Selective enamel ³	Total-etch
	•	•

Glass-ceramic



Etch
Hydrofluoric acid

•	•	•
Selective enamel ³	Total-etch	
•	•	

Resin based



Sandblast
Alumina 30 – 50 µm, 2 bar (30 PSI)

Clean
Alcohol

•	•	•	(•) ²
Selective enamel ³	Selective enamel ³	Total-etch	
•	•	•	

3M™ RelyX™ Universal
Resin Cement



¹Main recommendations for typical cases, other options apply. Before using the products described, please refer to the instructions for use.

²Priming not required for 3M™ RelyX™ Fiber Posts

³Optional for cut enamel

Ordering information

3M™ RelyX™ Universal Resin Cement

Item no.	Product information		
Intro Kit			
56968	4x 3M™ RelyX™ Universal Resin Cement syringes shades TR, A1, A3O, WO (3.4g each), 60x 3M™ RelyX™ Universal Micro Mixing Tips, 20 Elongation Tips, 1 vial 3M™ Scotchbond™ Universal Plus Adhesive (5 ml), 1x 3M™ Scotchbond™ Universal Etchant (3 ml), 25 blue etchant tips, 50 orange microbrushes, step-by-step guides		
Trial Kits			
	Each trial kit contains: 1x 3M™ RelyX™ Universal Resin Cement syringe (3.4g), 15x 3M™ RelyX™ Universal Micro Mixing Tips, 5 Elongation Tips, 1 vial 3M™ Scotchbond™ Universal Plus Adhesive (1.5 ml), 1x 3M™ Scotchbond™ Universal Etchant (3 ml), 25 blue etchant tips, 50 orange microbrushes, step-by-step guides		
56969	Shade TR	56970	Shade A1
Refills			
	Each refill contains: 1x 3M™ RelyX™ Universal Resin Cement syringe (3.4g), 15x 3M™ RelyX™ Universal Micro Mixing Tips, 5 Elongation Tips		
56971	Shade TR	56973	Shade A3O
56972	Shade A1	56974	Shade WO
Value Packs			
	Each value pack contains: 3x 3M™ RelyX™ Universal Resin Cement syringes (3.4g each), 45x 3M™ RelyX™ Universal Micro Mixing Tips, 15 Elongation Tips		
56977	Shade TR	56978	Shade A1
Mixing Tips			
56975	30x 3M™ RelyX™ Universal Micro Mixing Tips		
56976	15x 3M™ RelyX™ Universal Micro Mixing Tips with Elongation Tips		



3M™ Scotchbond™ Universal Plus Adhesive

Item no.	Product information		
Vial			
41293	Intro Kit: 1x 3M™ Scotchbond™ Universal Plus Adhesive vial (5 ml), 50 disposable brushes, 1x 3M™ Scotchbond™ Universal Etchant (3ml), 25 dispensing tips		
41294	Refill: 3M™ Scotchbond™ Universal Plus Adhesive vial (5 ml)		
	1x 5 ml vial	41295	3x 5 ml vial
Unit Dose			
	3M™ Scotchbond™ Universal Plus Adhesive Unit Dose L-Pops, 1x 3M™ Scotchbond™ Universal Etchant (3ml), 25 dispensing tips		
41297	Intro Kit: 50x L-Pop™	41299	200x L-Pop™
41298	100x L-Pop™	41304	400x L-Pop™

3M™ Scotchbond™ Universal Etchant Etching Gel

41263	2x 3M™ Scotchbond™ Universal Etchant (3ml each), 50 dispensing tips
-------	---



Solventum
Charnwood Campus, Bakewell Road
Loughborough, LE11 5TH, United Kingdom

Solventum 3M Healthcare Ireland Ltd
2 Cumberland Place, Fenian Street, Dublin 2,
D02 H0V5, Ireland

For more information,
visit [Solventum.com](https://www.solventum.com)

© Solventum 2026. Solventum and the S logo are trademarks of Solventum or its affiliates. 3M and the 3M logo are trademarks of 3M. All other trademarks are owned by their respective owners.