



Cavilon™

Skin Care Solutions

3M™ Cavilon™ No Sting Barrier Film

3M™ Cavilon™ No Sting Barrier Film is the original and only terpolymer based alcohol free barrier film* that helps prevent skin damage before it occurs.

Its unique formulation of polymers forms a sting-free, waterproof, protective coating that is breathable and transparent, allowing for continuous visualization and monitoring of skin during movement or position changes.



*of leading competitors in the market, based on disclosed ingredient information.



Durable

Fast drying¹, long lasting, waterproof and doesn't wash off², making it easy for clinicians to use.



Versatile

Helps protect skin from friction and abrasion³, an improvement over many creams, ointments and pastes that can increase friction at the skin surface.



Gentle

Alcohol free, sting free, fragrance free, preservative free and low dermatitis potential³.



Proven

Unique formulation supported by 80+ pieces of evidence.



Effective

Formulation helps to maintain a protective coating, plus it's sterile* and compatible with chlorhexidine gluconate (CHG)⁴, making it essential for vascular access site protection.

*Wands and wipes only

Ideal for routine protection



Indications for Use

Cavilon No Sting Barrier Film is a liquid intended for use as a film-forming product, that upon application to intact or damaged skin forms a long lasting waterproof barrier, which acts as a protective interface between the skin and bodily wastes, fluids, adhesive products, and friction and shear. It is intended as a primary barrier against irritation from body fluids. The product may be used on adults, children, and infants over one month of age.

Contraindications

Cavilon No Sting Barrier Film is NOT to be used:

1. As the only covering in situations that require dressing protection from bacterial contamination/penetration, e.g. intravenous therapy catheter sites and full or partial thickness wounds.
2. On infected areas of the skin.



Cavilon No Sting Barrier Film application tips sheet

General direction for use:

- 1** Skin should be clean and dry prior to application of Cavilon No Sting Barrier Film.
- 2** **Foam applicator or wipe:** apply a uniform coating of film over the entire area of concern.
Spray: hold spray nozzle four to six inches from skin and apply a smooth, uniform coating of film over the entire area of concern.
- 3** If an area is missed, reapply to that area only after first application of Cavilon No Sting Barrier Film has dried (approximately 30 seconds). In situations where exposure to caustic irritants is expected, a second coating of the film can be applied to the skin after the first coating has dried completely.
- 4** If Cavilon No Sting Barrier Film is applied to the area with skin folds or other skin to skin contact, make sure that skin contact areas are separated to allow the film to thoroughly dry before returning to the normal position.
- 5** **When used under adhesive tapes, dressings or devices:**
 - a) Allow Cavilon No Sting Barrier Film to thoroughly dry before covering with dressings or adhesive products.
 - b) Reapplication of Cavilon No Sting Barrier Film is necessary each time dressing and/or adhesive products are changed; the barrier film is removed by the adhesive.
- 6** **When used as a protectant against body fluids, feces, or urine etc. and no adhesive products are applied to skin:**
 - a) Reapplication of Cavilon No Sting Barrier Film is recommended every 24–72 hours, depending on frequency of cleansing.
 - b) In extreme cases (e.g. constant diarrheal stooling) with very frequent cleansing, more frequent applications **may** be necessary (i.e. every 12–24 hours).
- 7** If desired, the film can be removed by using a medical adhesive remover as directed. Clean and dry the involved area and then reapply Cavilon No Sting Barrier Film.
- 8** Replace cap over spray nozzle when not in use to prevent accidental dispensing.



3M Company
2510 Conway Ave
St. Paul, MN 55144 USA

Phone 1-800-228-3957
Web 3M.com/medical

References

1. 3M data on file. TEAM MISC 05 001563, CLIN RPT FINAL INV US 05 289804 (dry time). 2. 3M data on file. CLIN RPT FINAL ICH2 US 05 291160. 3. Campbell K, Woodbury MG, Whittle H, Labate T, Hoskin A. A clinical evaluation of 3M™ Cavilon™ No Sting Barrier Film. *Ostomy Wound Management*. 4. 3M data on file. TEAM MISC 05 005732 and SPONSOR FINAL RPT 05 002049.

© 2022 3M. All rights reserved. 3M and the other marks shown are marks and/or registered marks. Unauthorized use prohibited. Used under license in Canada. All other marks are property of their respective owners. 70-2013-1257-9