

# 3M™ Promogran Prisma™ Rope Collagen Matrix with ORC and Silver

Promogran Prisma Rope 3/8" x 3/8" x 12-5/8"

Intended for the management of exuding wounds with characteristics including deep tunneling or undermining. A rope dressing provides a convenient solution for managing these wounds.

## Prepare your wound

Prepare wound according to your standard wound care protocol or health care provider instruction.

## Apply your dressing (see reverse side)

1. Peel back the tray cover and leave dressing in tray until ready to apply.
2. For a small or deep exuding wound, cut the dressing prior to application or pack dressing into the wound.
3. If the wound bed is wet, apply the dressing directly.
4. For a wound with low or no exudate, apply Promogran Prisma Rope Matrix and hydrate with saline solution. This will initiate the transformation of the Promogran Prisma Rope Matrix into a gel.
5. Cover and secure the dressing with the materials provided:

---

## Need to re-order more Promogran Prisma Rope Matrix?

See below for a table of manufacturer item codes and available sizes.

### 3M™ Promogran Prisma™ Rope Collagen Matrix with ORC and Silver

Item code	Size	Eaches/Carton/Box	HCPCS code
MA032	3/8" x 3/8" x 12 5/8"	6 ea/ct - 4 ct/bx	A6024

\*This user guide for dressing application and removal is intended to be prepared/completed by the patient's treating clinician. Please refer to your health care provider who prescribed Promogran Prisma Rope Matrix if you require additional information on your dressing schedule and plan of care.



## 1. Dressing preparation

- Peel back the tray cover and leave dressing in tray until ready to apply.



## 2. Dressing application

- Center the dressing over the wound and apply the dressing.
- For a small or deep exuding wound, cut the dressing prior to application or pack dressing into the wound.
- For a wound with low or no exudate, apply the dressing and hydrate with saline solution. This step will initiate the transformation of the 3M™ Promogran Prisma™ Rope Collagen Matrix with ORC and Silver into a gel.
- Dressing should remain in the wound and not on the skin edges.



## 3. Secure dressing application

- Cover and secure the dressing with the materials provided:

---

---

---



## 4. Dressing reapplication

(follow your doctor's instructions)

- Reapply daily.
- Reapply every other day.
- Do not allow Promogran Prisma Rope Matrix to dry out.
- Other: \_\_\_\_\_

---

Contact your local Solventum representative, call the Solventum Helpline at **800-228-3957**, or visit **Solventum.com** for more information.



**Solventum Medical Surgical**

**Phone** 800-228-3957  
**Web** Solventum.com

**Note:** Specific indications, contraindications, warnings, precautions and safety information exist for these products and therapies. Please consult a clinician and product instructions for use prior to application. Rx only.

© Solventum 2025. Solventum and the S logo are trademarks of Solventum or its affiliates. 3M is a trademark of 3M. All other trademarks are owned by their respective owners.

US\_70-2013-1387-4