


3M™ Nasogastric Securement Device

Application and removal guide



3M™ Nasogastric Securement Device application

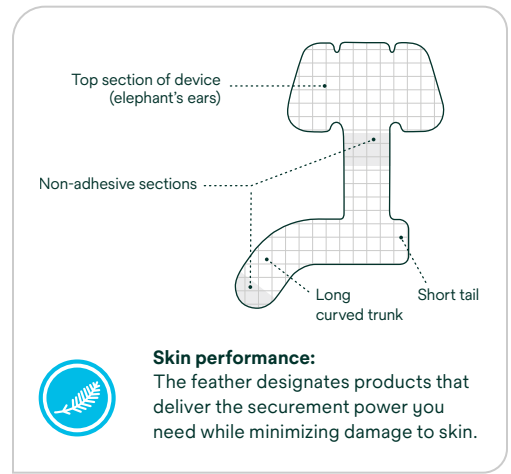


Select device based on tubing size: 1501U for 10-18Fr; 1500U for 8-12Fr.

1. Prep the skin and tubing

- Thoroughly clean/scrub nose and tubing with an alcohol wipe and allow to dry.
- If using any skin preps, allow to dry completely.


Note: Clean starting surfaces are critical to successful adhesion to skin and to tubing.

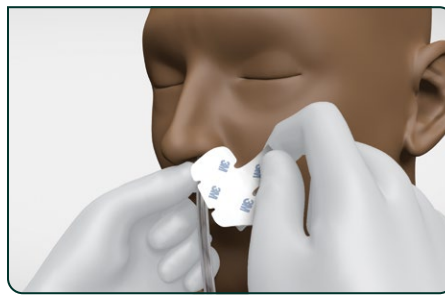

2. Remove device top liner


- Open package.
- Remove the top liner from the device, exposing the adhesive on the top of the device.

Note: Keep tube stabilized throughout duration of securement.

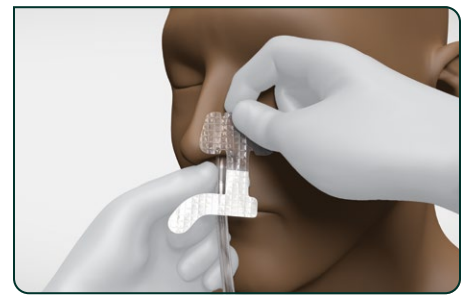

3. Align & apply top of device


- Center the top of device over naris on the side with the tube.
- Align top of non-adhesive section with base of naris.
- Place the top portion onto the nose fitting into the natural crease.
- Press firmly and massage to activate adhesion to skin.

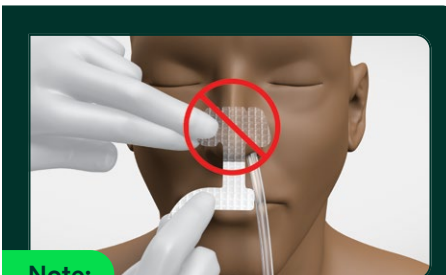

Note:

Do NOT center over nose

Off-center placement and the non adhesive section allow the tube to hang freely to help minimize contact with the naris.

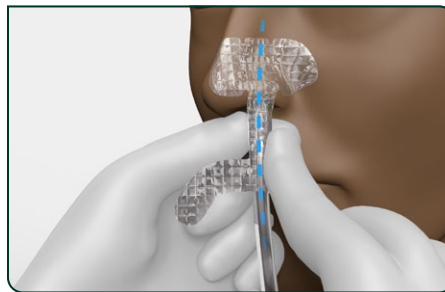

4. Begin bottom placement

- Remove bottom liner exposing adhesive
- Identify the short tail (extending right) and long curved trunk (extending left) since must apply short tail first.
- Align the tube with vertical part of device and press onto straight tube.
- Wrap the short tail around the back of the tube, careful to avoid adhesive-to-adhesive contact with the long trunk.




5. Wrap tube

- Spiral the curved tail around the tube in a spiral fashion, overlapping 50/50 to maximize adhesion to tubing.
- Press firmly on the device to activate adhesion to tubing, pinching around the tubing top to bottom.
- A non-adhesive section at the end of the tail is intended to help aid removal.


Note: Do not wrap the tail of the 3M™ Nasogastric Securement Device too tightly or closely to the naris as this could lead to medical device-related pressure injury.

3M™ Nasogastric Securement Device maintenance and tips

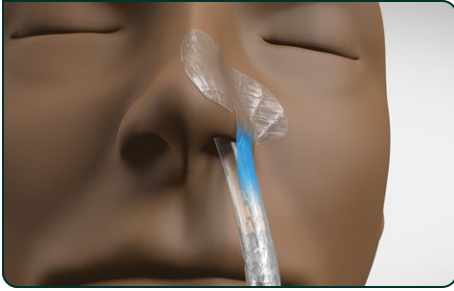

Non-adhesive feature

- A non-adhesive section at the base of the nose is intended to help the tube hang freely.
- Do not attempt to tape over the non-adhesive section of the device.

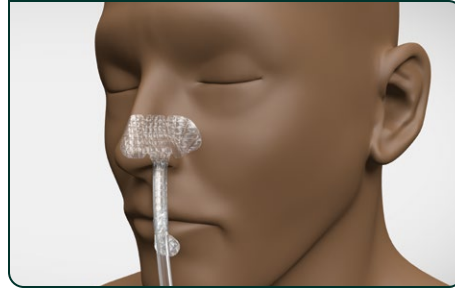

Monitor

- The site should be observed regularly for signs of skin complications or tube migration per facility protocol.
- Skin complications may be signaled by redness, color changes, skin that is abnormally warm or cool to the touch, non-blanching skin on palpation, pain, tingling, or itching.
- If skin irritation or injury is suspected, remove the device, inspect the site directly, and determine appropriate medical intervention.

Device replacement



1. Unwrap

- While stabilizing the tube, use the non-adhesive tab at the end of the tail to unwrap the device from the tube.




2. Remove and replace

- Remove the adhesive device from the nose while stabilizing the tube.
- Once the device is removed, a new one may be applied.

Note: Peel low and slow while removing device from nose.

Note: Ensure the skin and tube are clean, dry, and free of residue and skin oil before application of a new securement device.

Device removal

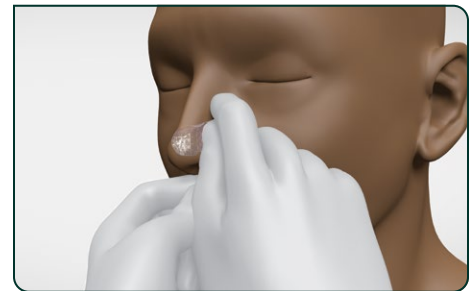

Discontinuing NG therapy

- Remove the securement device from the nose.
- Remove the tube from the patient.

Note: Peel low and slow.

Alcohol wipes may be used to remove any residue.

Avoid use of adhesive removers for any adhesive residue left on the skin or tubing.



Refer to product Instructions for Use for other important information.



Scan the QR code to view a product training video, or visit [Solventum.com](https://www.solventum.com) to learn more.



Solventum Corporation
1750 Yankee Doodle Road
Eagan, MN 55121
USA

Phone 800-228-3957
Web [Solventum.com](https://www.solventum.com)

© Solventum 2026. All Rights Reserved. Solventum and the S logo are trademarks of Solventum or its affiliates. 3M and the 3M logo are trademarks of 3M. Other trademarks are the property of their respective owners.

GL_ENG_70-2013-1311-4