


3M™ Tube Securement Device

Application and removal guide

It is okay if you find another way to apply the device that is more comfortable to you, as long as the applied device is fully wrapped around the tube, lifted off the skin, and has been pressed firmly into place.

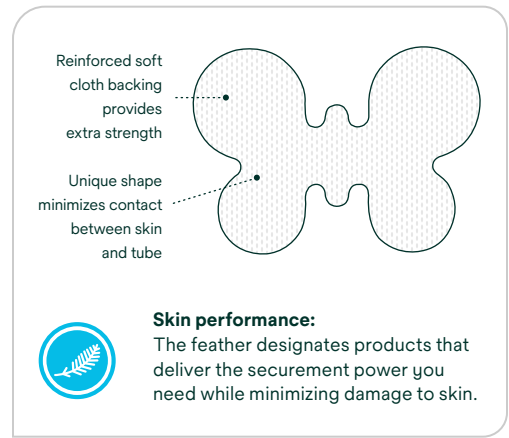

3M™ Tube Securement Device Application




1. Prepare

- Select the device size for application: 1501U fits 12-24 French; 1500U fits 6-12 French.
- Thoroughly clean skin and tubing with alcohol wipes.
- Ensure skin and tubing are clean, dry and completely free of residue.
- Open package.
- Completely remove the device from line.



2. Apply first wing

- While minimizing tube manipulation, apply one wing (half) of the device onto the patient's skin.
- Note:** Do not stretch device during application as tension may result in skin trauma.

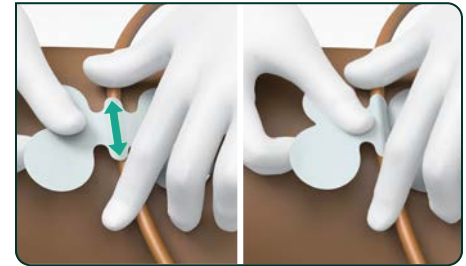

3. Secure tubing

- Align the tube along the center "body" of the device.
- Fully wrap the device around the tube.




4. Begin bottom placement

- Apply the other wing (half) of the device onto the patient's skin.
- Apply firm pressure on the device, including the edges, to enhance the adhesion to the skin and to the tubing.



5. Check final positioning


- Ensure the tube is lifted slightly from the skin to reduce the risk of pressure injury.

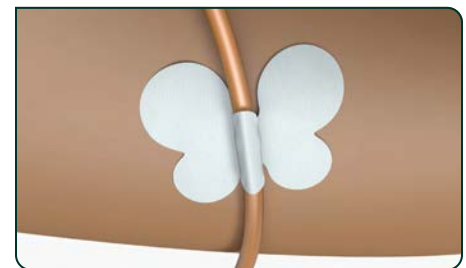

Alternate method

- There is another method to apply the device after selecting the right size device and cleaning the skin and tubing. Importantly, this achieves the device fully wrapped around tubing, lifting tubing off the skin.
- Align the body (center) of the device to the tubing.
- Wrap both wings around the tubing so it is lifted off the skin.
- Attach both wings to the skin.
- Apply firm pressure to the skin and tubing to enhance adhesion.

3M™ Tube Securement Device Replacement or Removal

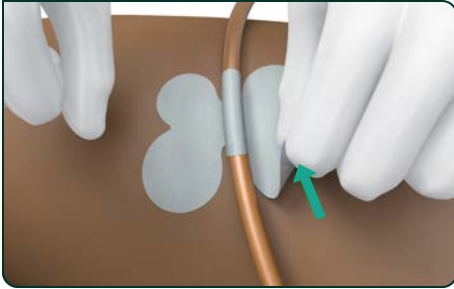

1. Initiate removal

- Minimize tube manipulation during removal.
- Peel back one corner of the device.



2. Remove first wing

- Stabilize the tube to the patient.
- Keep device parallel to skin.
- Completely remove one wing.

Note: Peel low and slow.



3. Stabilize and remove

- Carefully peel the device off the tube.
- Remove the remaining wing of the device from the skin.

Note: Peel low and slow.

If applying a new device, clean the skin and tubing with alcohol and restart application steps.



Refer to product
Instructions for Use for other
important information.



Scan to view a product training video or visit
[3M.com/TubeSecurement](https://www.3m.com/TubeSecurement) to learn more.



Solventum Medical Surgical
1750 Yankee Doodle Road.
Eagan, MN 55121

Phone 800-228-3957
Web [Solventum.com](https://www.solventum.com)

© Solventum 2026. All Rights Reserved. Solventum and the S logo are trademarks of Solventum or its affiliates. 3M and the 3M logo are trademarks of 3M. Other trademarks are the property of their respective owners.

US_70-2013-1330-4