




**AbThera™**  
Open Abdomen Therapy

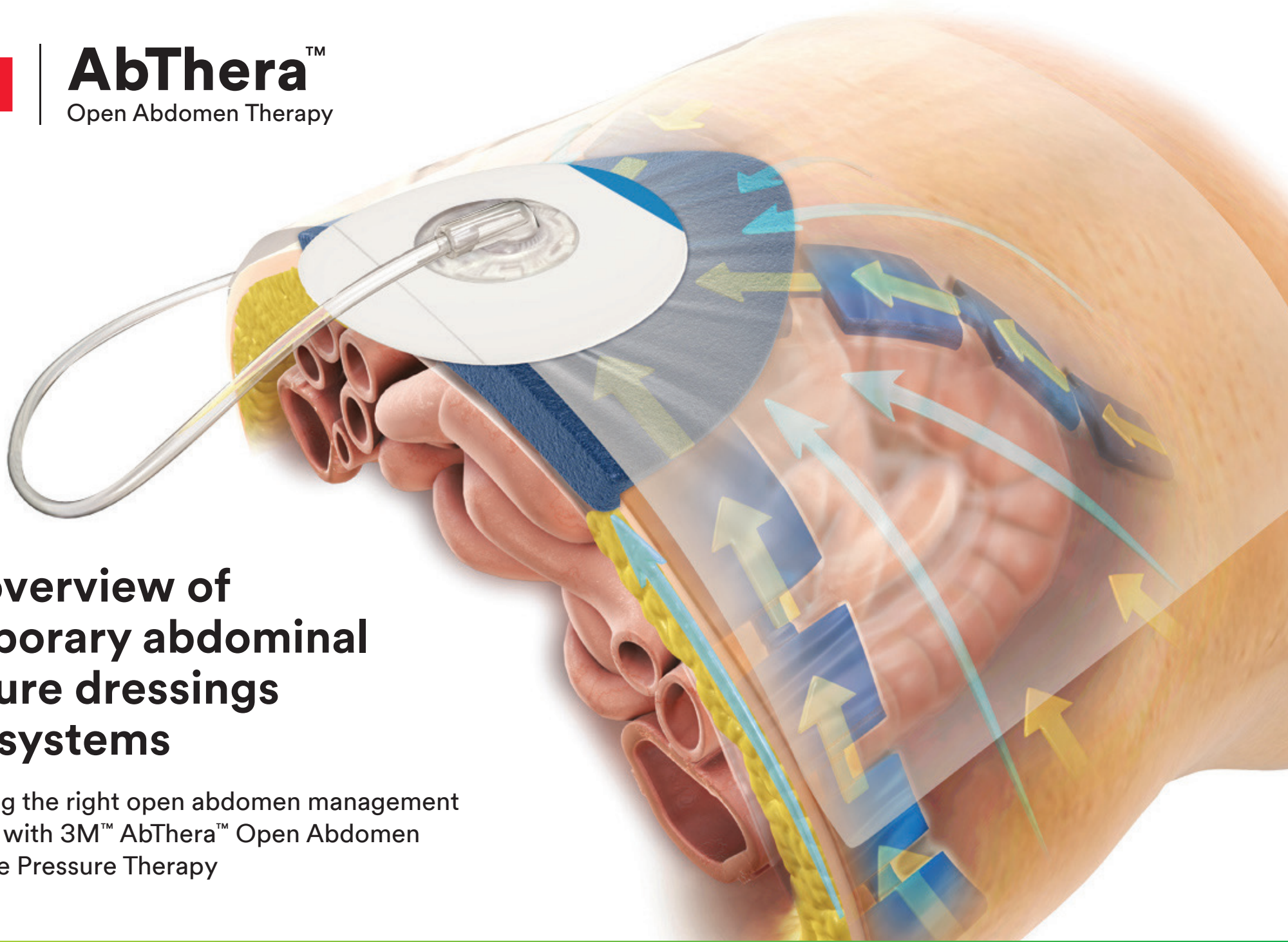

## An overview of temporary abdominal closure dressings and systems

Choosing the right open abdomen management solution with 3M™ AbThera™ Open Abdomen Negative Pressure Therapy

# Temporary abdominal closure overview

Temporary abdominal closure (TAC) methods allow for stabilisation of the patient to better endure subsequent operations. The abdomen is left open at the time of operation to facilitate re-exploration after trauma, allowing the abdomen to be accessible for washouts, and to stabilize the patient for further surgery.<sup>1</sup>

The method of a temporary abdominal closure may play an important role in positive clinical and economic outcomes.<sup>2</sup>



## What the ideal TAC should be able to do<sup>3</sup>

- ▶ Limit mortality
- ▶ Support a high rate of closure
- ▶ Limit complications
- ▶ Limit dressing changes
- ▶ Protect the fascia and skin
- ▶ Minimise loss of domain prevent the development of ACS
- ▶ Limit contamination
- ▶ Be easily applied
- ▶ Prevent adhesions
- ▶ Allow room for abdominal contents to expand
- ▶ Decrease bowel edema
- ▶ Allow for evacuation of fluids

## The importance of Primary Fascial Closure<sup>3</sup>

Patients in whom early definitive primary closure cannot be performed are more likely to experience:

- ▶ Sepsis
- ▶ Increased ICU and hospital LOS
- ▶ Enteroatmospheric fistulas
- ▶ Incisional hernia

Patients with fascial closure within 4–7 days are associated with:

- ↓ lower mortality
- ↓ and fewer complications


# Methods of temporary abdominal closure<sup>4</sup>



## Skin-only closure

A basic temporary abdominal closure method that involves using the skin to provide some abdominal wall stability.

Up to 30 surgical clips (1cm apart from each skin edge) are utilised to perform a skin-only closure.

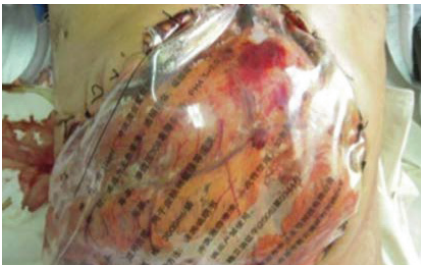

## Bogota Bag

Technique involves cutting a previously sterilised IV bag into an open, oval shape and suturing it to the skin.




## Barker's vacuum pack technique

A technique that consists of multi-component layers from common materials available in most hospitals. It utilises:


- ▶ A non-adherent polyethylene sheet (must be manually fenestrated)
- ▶ A moist surgical towel cover
- ▶ Two silicone drains over the towels
- ▶ An adhesive sheet

Continuous wall suction is applied to remove fluid.

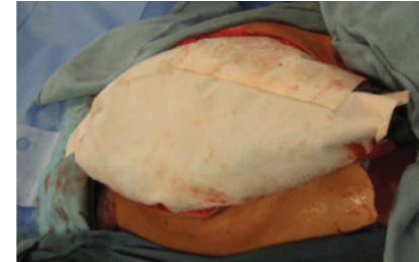

## Wittmann Patch™

Consists of two detachable components: a loop sheet and a closure sheet sutured to the abdominal fascia. Closure is achieved by overlapping, and sequentially tightening the sheets.

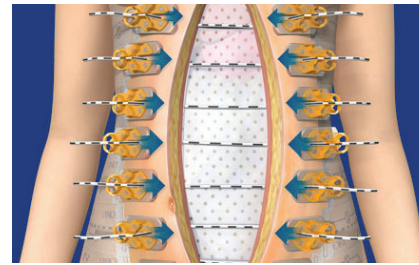

## ABRA® Abdominal<sup>5</sup>

A re-approximation anchor system intended to retract the abdominal wall defect by cyclic stretching of elastomer bands running across the wound opening.

## Commercial NPWT Open Abdomen Dressing

All inclusive dressing kits specifically designed for use in the open abdomen with negative pressure therapy like the 3M™ AbThera™ Advance Open Abdomen Dressing.



# 3M™ AbThera™ Advance Open Abdomen Dressing components

## 3M™ AbThera™ Fenestrated Visceral Protective Layer

Provides separation between abdominal wall and viscera, protecting abdominal contents. It features six encapsulated foam arm extensions that aid in fluid removal and negative distribution deep in the paracolic gutters.

## 3M™ SensaT.R.A.C.™ Technology

Our proprietary SensaT.R.A.C. Technology provides a real-time pressure feedback system and adjusts and monitors pressure at the abdomen.

3M™ AbThera™ Open Abdomen Negative Pressure Therapy is the only open abdomen dressing designed to work with 3M™ V.A.C.® Therapy and proprietary SensaT.R.A.C. Technology.



## 3M™ AbThera™ Advance Perforated Foam

Under negative pressure, the unique configuration of the AbThera Advance Perforated Foam is designed to collapse medially while maintaining its vertical rigidity.

## 3M™ V.A.C.® Drape

Provides a closed system to help isolate and protect abdominal contents from the external environment.

3M™ AbThera™ Advance Open Abdomen Dressing is the only temporary abdominal closure dressing that features a foam with cutouts configured to draw the wound edges together.

# Comparing TAC techniques and products<sup>4</sup>

**3M™ AbThera™ Advance  
Open Abdomen Negative  
Pressure Therapy**



What does ◊ refer to?

RENASYS®  
AB

Suprasorb®  
CNP

VivanoMed®  
Abdominal  
Kit

Invia®  
Abdominal  
Dressing Kit

Barker's  
Vacuum Pack  
Technique

Wittmann  
Patch™

ABRA®  
Abdominal

Bogota  
Bag

	3M™ AbThera™ Advance Open Abdomen Negative Pressure Therapy	RENASYS® AB	Suprasorb® CNP	VivanoMed® Abdominal Kit	Invia® Abdominal Dressing Kit	Barker's Vacuum Pack Technique	Wittmann Patch™	ABRA® Abdominal	Bogota Bag
Provides medial tension	◊	•	•	•	•	•			
Fluid removal	◊	•	•	•	•	•			
Protects the skin and fascia	◊	•	•	•	•	•			
No sutures or staples required	◊	•	•	•	•	•			
Ability to monitor fluid output	◊	•	•	•	•	•			
Visceral Protective Layer with encapsulated foam	◊								
Perforated foam with cutouts designed to collapse medially	◊								

## Pre-clinical evidence

*No correlation to human use*

In a comparative study of four healthy pigs with an open abdominal wound that were treated with either 3M™ AbThera™ SensaT.R.A.C.™ Open Abdomen Dressing or 3M™ AbThera™ Advance Open Abdomen Dressing at -125mmHg for 5 minutes, results showed that the AbThera Advance Dressing showed a difference in the following:

# 31%

increase in overall  
tissue movement<sup>6</sup>  
(N=82, p<0.05)

# 39%

increase in skin movement<sup>6</sup> (N=42, p<0.05)

# 20%

increase in fascia movement<sup>6</sup> (N=40, p<0.05)


In this study, no change in intra-abdominal pressure was observed when negative pressure was applied.

# 3M™ AbThera™ Open Abdomen Negative Pressure Therapy evidence



## Decrease in all-cause mortality

In two separate studies, when compared to Barker's vacuum pack technique, AbThera Therapy demonstrated greater reduction in 30-day and 90-day all-cause mortality.<sup>2,7</sup>

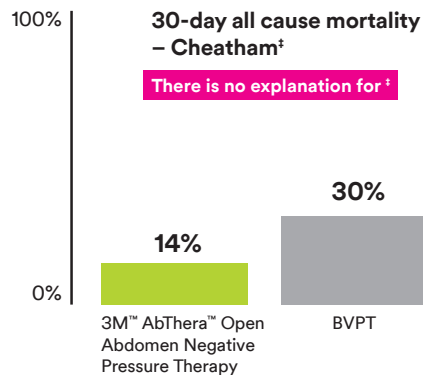


### Prospective Study Examining Clinical Outcomes Associated with a Negative Pressure Wound Therapy System and Barker's Vacuum Packing Technique

Cheatham ML, Demetrios D, Fabian TC, Kaplan MJ, et al.

Of 280 patients enrolled from 20 study sites in the U.S., 168 patients received at least 48 hours of consistent Temporary Abdominal Closure (TAC) therapy (111 AbThera Therapy, 57 BVPT).

- ▶ **Thirty-day PFC rate:** 69% for AbThera Therapy and 51% for BVPT ( $p = 0.03$ )
- ▶ **Thirty-day all-cause mortality:** 14% for AbThera Therapy and 30% for BVPT ( $p = 0.01$ )
- ▶ **Median of days to PFC:** 9 days for AbThera Therapy vs 12 days for BVPT ( $p = 0.12$ )

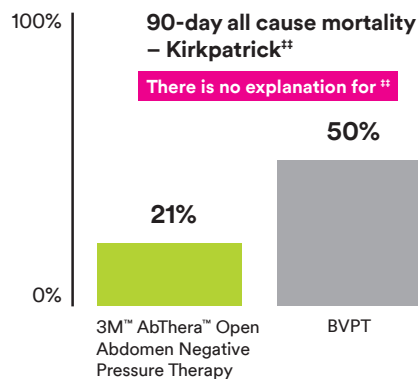

### Active Negative Pressure Peritoneal Therapy After Abbreviated Laparotomy: The Intraperitoneal Vacuum Randomized Controlled Trial

Kirkpatrick AW, Roberts DJ, Faris PD, et al.


A total of 45 adults with abdominal injuries (46.7%) or intra-abdominal sepsis (52.3%) were randomly allocated to AbThera Therapy ( $n = 23$ ) or Barker's vacuum pack ( $n = 22$ ). Primary endpoint to identify the difference in plasma concentration of interleukin-6 at 24- and 48-hours after application were not met.

- ▶ **90-day mortality:** 50% for Barker's vacuum pack and 21.7% for AbThera Therapy ( $p = 0.04$ )



### Increase in primary fascial closure

In two separate studies, when compared to Barker's vacuum pack technique, 3M™ AbThera™ Open Abdomen Negative Pressure Therapy resulted in an increase in primary fascial closure.<sup>2,8</sup>

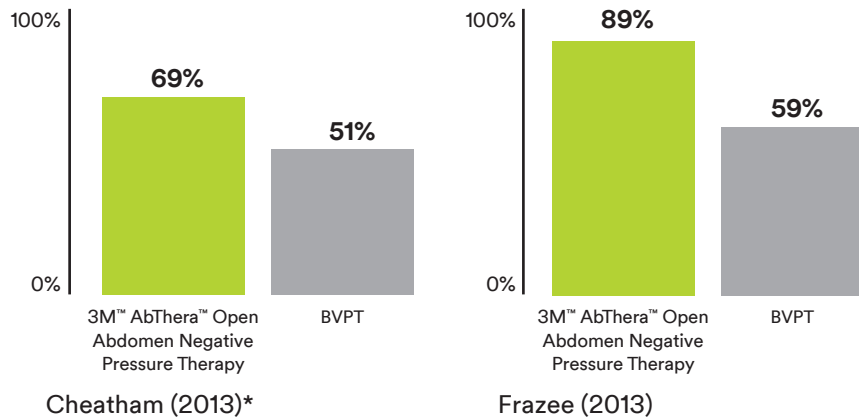

### Are commercial negative pressure systems worth the cost in open abdomen management?

Frazee RC, Abernathy SW, Jupiter DC, et al.

Thirty-seven open abdomen patients who had temporary abdominal closure with the AbThera Therapy device were compared with 37 open abdomen patients managed with the Barker's technique.

#### Ultimate midline fascial closure:

89% (33/37) for AbThera Therapy and 59% (22/37) Barker's Vacuum Packing Technique (BVPT) ( $p < 0.05$ )


\*Of 280 patients enrolled from 20 study sites in the U.S., 168 patients received at least 48 hours of consistent Temporary Abdominal Closure (TAC) therapy (111 AbThera Therapy, 57 BVPT). Median of days to PFC were 9 days for AbThera Therapy vs 12 days for BVPT ( $p = 0.12$ ). Thirty-day PFC rate was 69% for AbThera Therapy and 51% for BVPT ( $p = 0.03$ ). Thirty-day all-cause mortality was 14% for AbThera Therapy and 30% for BVPT ( $p = 0.01$ ).



### Decrease in resource utilisation<sup>9</sup>

In a 42-patient study, AbThera Therapy and BVPT were compared for resource utilisation. 32 patients received AbThera Therapy and 12 Barker's Vacuum Packing Technique (BVPT). AbThera Therapy showed clinically significant trends towards decreased resource utilisation.

	3M™ AbThera™ Open Abdomen Negative Pressure Therapy	BVPT
Hospital days ( $p = 0.17$ )	20	31
ICU days ( $p = 0.1$ )	11	17
Ventilator days ( $p = 0.19$ )	9	13
# of dressing changes ( $p = 0.047$ )	2	3

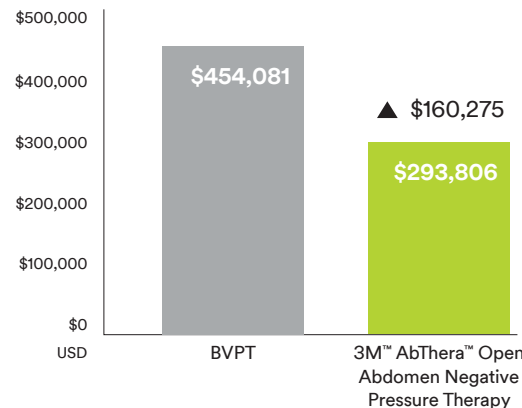

### 3M™ AbThera™ Open Abdomen Negative Pressure Therapy demonstrated a decrease in:

- ▶ ICU days
- ▶ Ventilator days
- ▶ Hospital days
- ▶ Days to abdominal closure
- ▶ Hospital charges

**Patients who received 3M™ AbThera™ Open Abdomen Negative Pressure Therapy reduced hospital charges per patient by \$160,275 USD.**

# Ordering information

## Item number


### ABT1055

(Includes 3M™ AbThera™ Fenestrated Visceral Protective Layer, (2) 3M™ AbThera™ Advance Perforated Foam, (4) 3M™ V.A.C.® Drapes, and 3M™ SensaT.R.A.C.™ Pad and Tubing)

For use with negative pressure therapy provided by the 3M™ V.A.C.® Ulta Therapy Unit.

## Qty.

5 per case

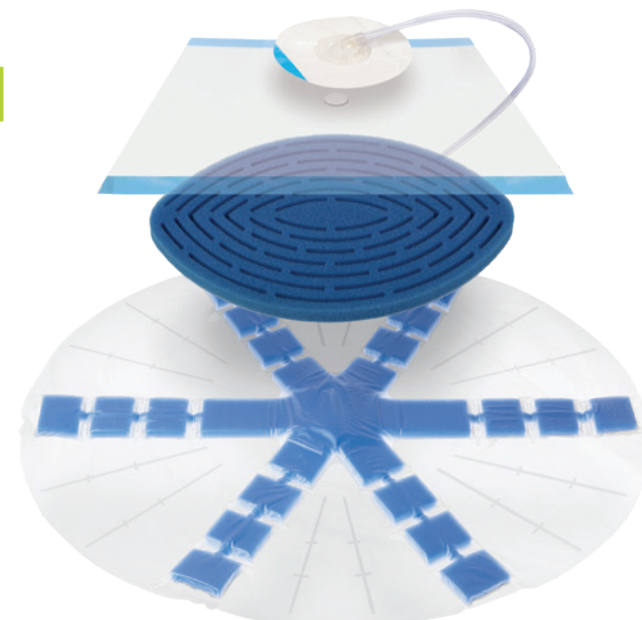

## References

1. Franklin ME, Alvarez A, Russek K. Negative Pressure Therapy: A Viable Option for General Surgical Management of the Open Abdomen. *Surgical Infections* 2012 December 1;19(4):353-63.
2. Cheatham ML, et al. Prospective Study Examining Clinical Outcomes Associated with a Negative Pressure Wound Therapy System and Barker's Vacuum Packing Technique. *World Journal of Surgery*. 2013 Sep;37(9):2018-30.
3. Fitzpatrick ER. Open abdomen in trauma and critical care. *Critical care nurse*. 2017 Oct 1;37(5):22-45.
4. Huang Q, Li J, Lau WY. Techniques for Abdominal Wall Closure after Damage Control Laparotomy: From Temporary Abdominal Closure to Early/Delayed Fascial Closure-A Review. *Gastroenterol Res Pract*. 2016;2016:2073260. doi:10.1155/2016/2073260
5. Dynamic Tissue Systems. ABRA® Abdominal. Instructions for Use. Accessed June 2019.
6. Schmidt M, Hall C, Mercer D, Kieswetter K. Novel Foam Design Actively Draws Wound Edges Together Under Negative Pressure: Benchtop and Pre-clinical Assessment. Presented at: Society of Advanced Wound Care / Wound Healing Society; November 3, 2018; Las Vegas, Nevada.
7. Kirkpatrick AW, Roberts DJ, Faris PD et al. Active Negative Pressure Peritoneal Therapy After Abbreviated Laparotomy: The Intraperitoneal Vacuum Randomized Controlled Trial. *Ann Surg* 2015;262:38-46.
8. Frazee RC, Abernathy SW, Jupiter DC, et al. Are commercial negative pressure systems worth the cost in open abdomen management? *J Am Coll Surg* 2013;216:730-735.
9. Safcsak K, Cheatham ML. ABTHERA™ Open Abdomen Negative Pressure System versus Barker's Vacuum Pack Technique: analysis of resource utilization. Poster presented at the Fifth World Congress on the Abdominal Compartment, Orlando, FL. August 10-13, 2011.



## KCI Medical Ltd.

3M Centre, Cain Road,  
Bracknell, RG12 8HT, UK  
0800 980 8880  
UKCUSTSERV@kci-medical.com  
3M.com/KCI

## KCI Medical Ltd.

Unit 6-7, 2050 Orchard Avenue  
City West Business Campus, Citywest,  
Co. Dublin, D24 X897, Ireland  
1800 33 33 77  
IRCS@kci1.com

**Note:** Specific indications, contraindications, warnings, precautions and safety information exist for these products and therapies. Please consult a clinician and product instructions for use prior to application. This material is intended for healthcare professionals.

Wittmann Patch is a trademark of Starsurgical, Inc. ABRA Abdominal is a trademark of Dynamic Tissue Systems. Renasys AB is a trademark of Smith & Nephew, Inc. Invia® Abdominal Dressing Kit is a trademark of Medela AG.

© 2023 3M. All rights reserved. 3M and the other marks shown are marks and/or registered marks. Unauthorised use prohibited. OMG892610