



# Basic application guide for 3M<sup>™</sup> V.A.C.<sup>®</sup> Dressing

For wounds without exposed vessels, organs,  
tendons and nerves

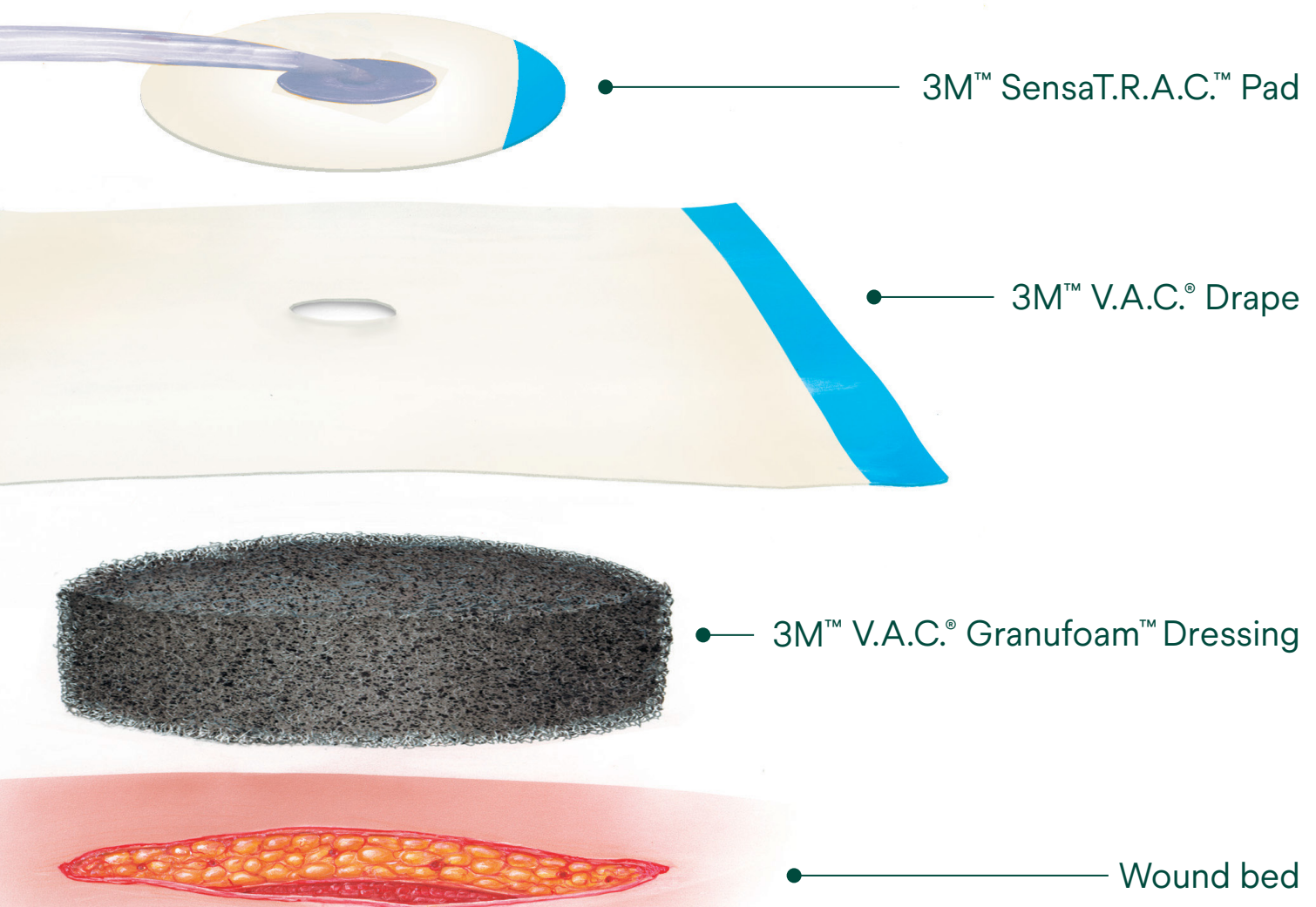


# Basic application guide for 3M™ V.A.C.® Dressing

## Step 1: Assess wound size and type

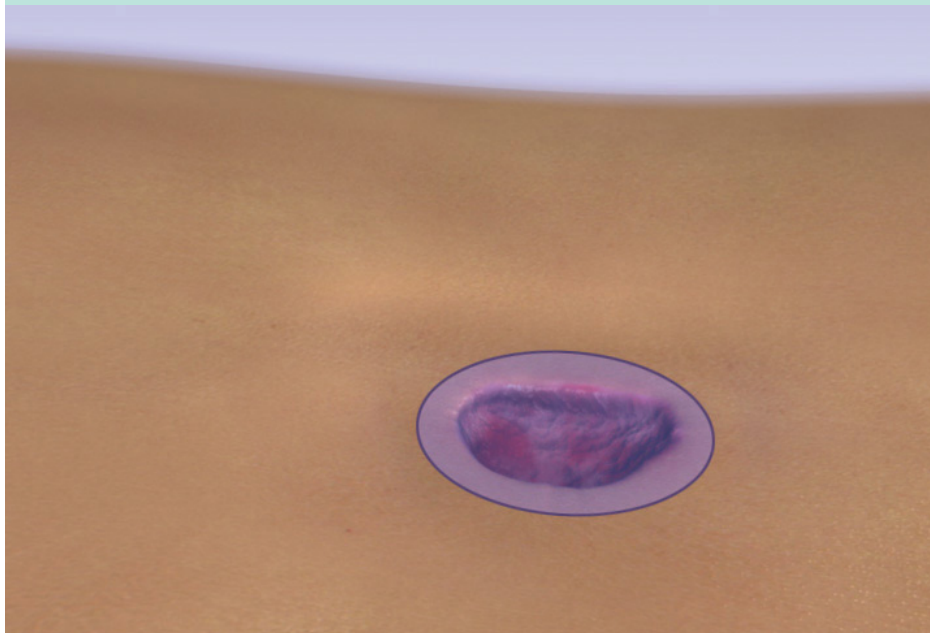

Prepare wound bed appropriately.


## Step 2: Cut appropriate dressing size

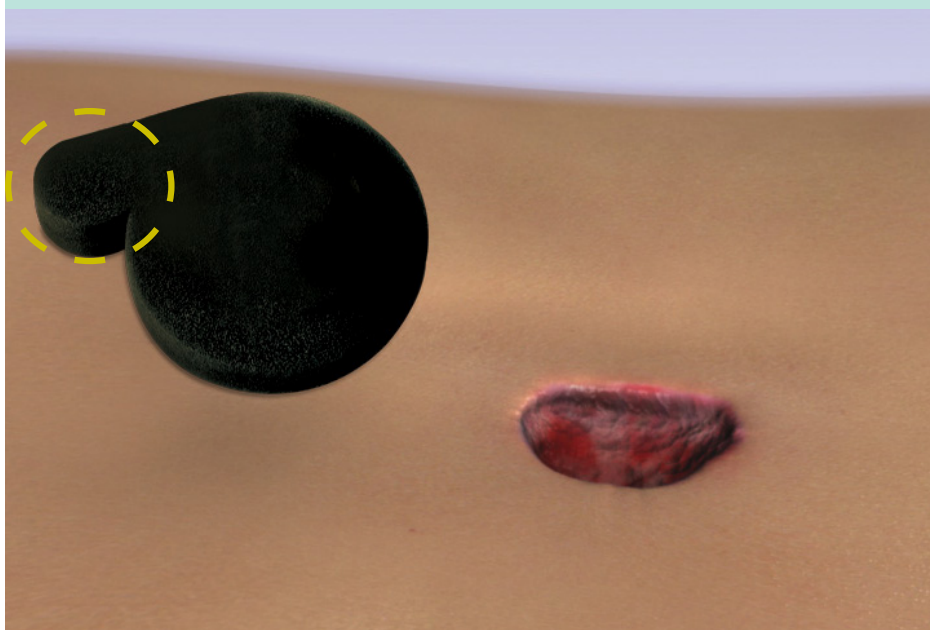

Do not cut dressing over wound as this could result in foam particles falling into the wound.


# Basic application guide for 3M™ V.A.C.® Dressing

## Step 3: Place dressing in wound

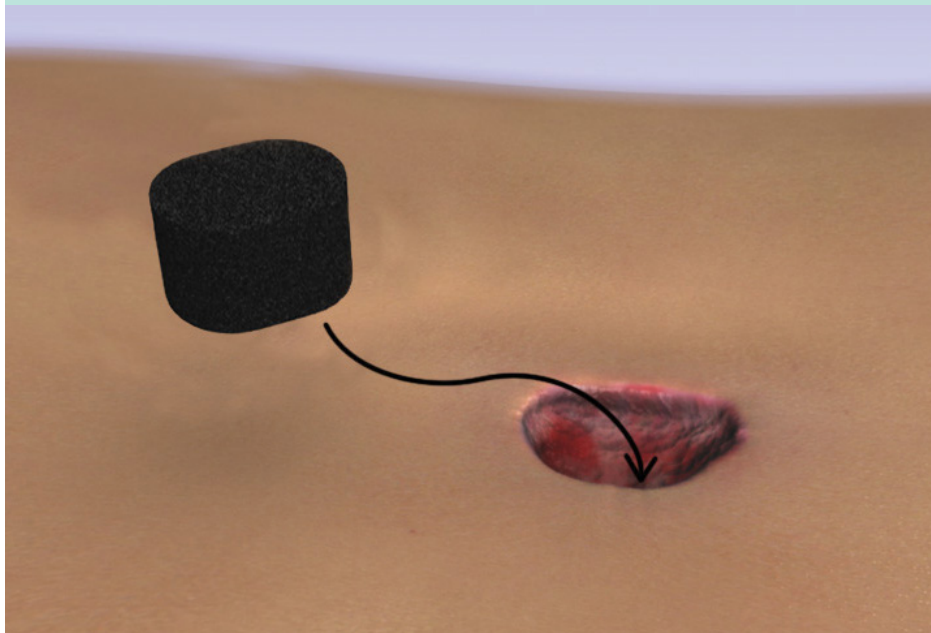

Note the total number of pieces of foam used in the wound.


## Step 4: Trim 3M™ V.A.C.® Drape to size

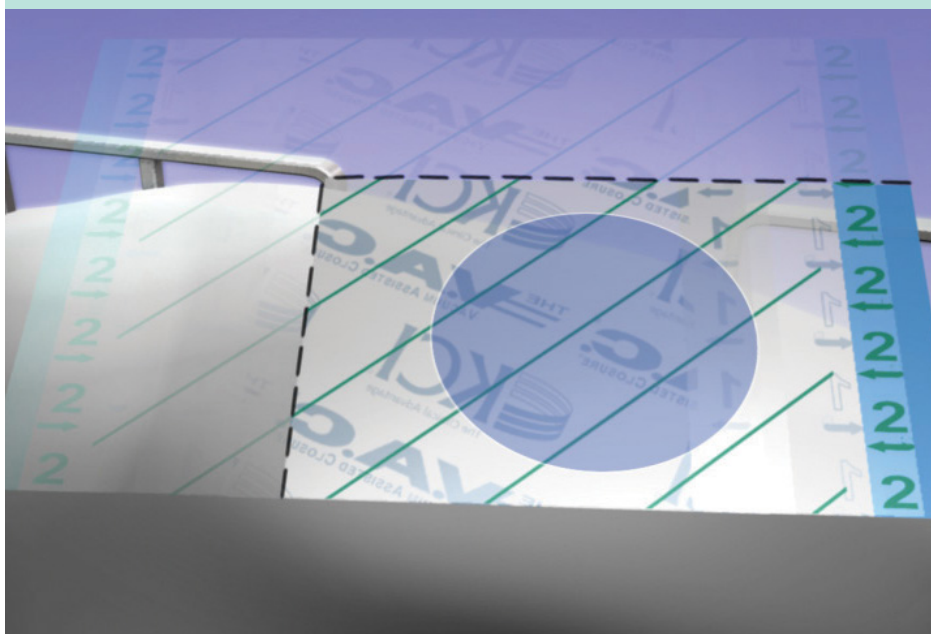

Peel back one side of Layer 1 and place adhesive side down over wound. Remove the remaining side of Layer 1, the green striped stabilisation Layer 2 and the perforated blue handing tab.


# Basic application guide for 3M™ V.A.C.® Dressing

## Step 5: 3M™ SensaT.R.A.C™ Pad application

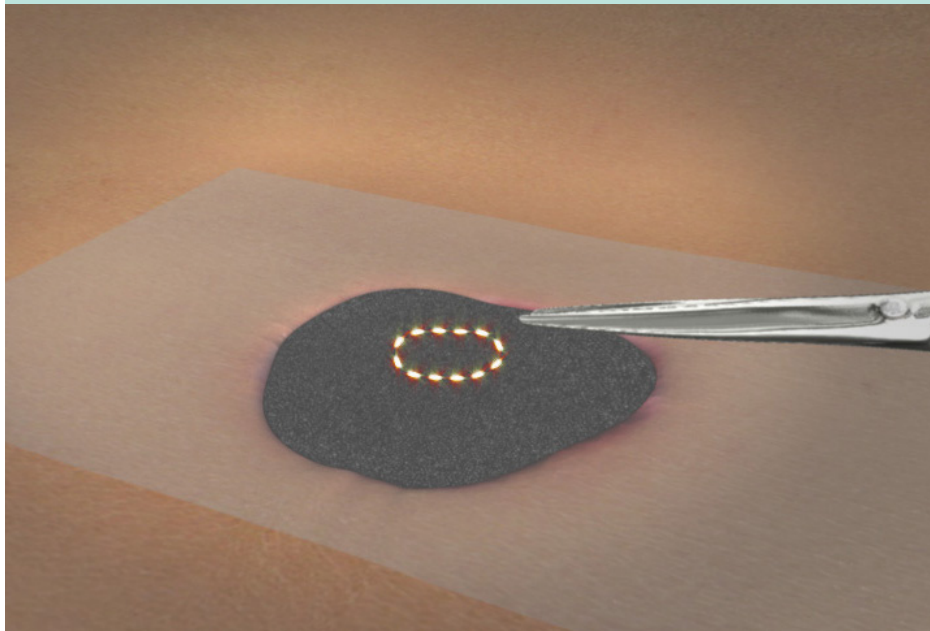

Identify application site for SensaT.R.A.C. Pad.


## Step 6: Pinch 3M™ V.A.C.® Drape and cut a 2.5 cm round hole

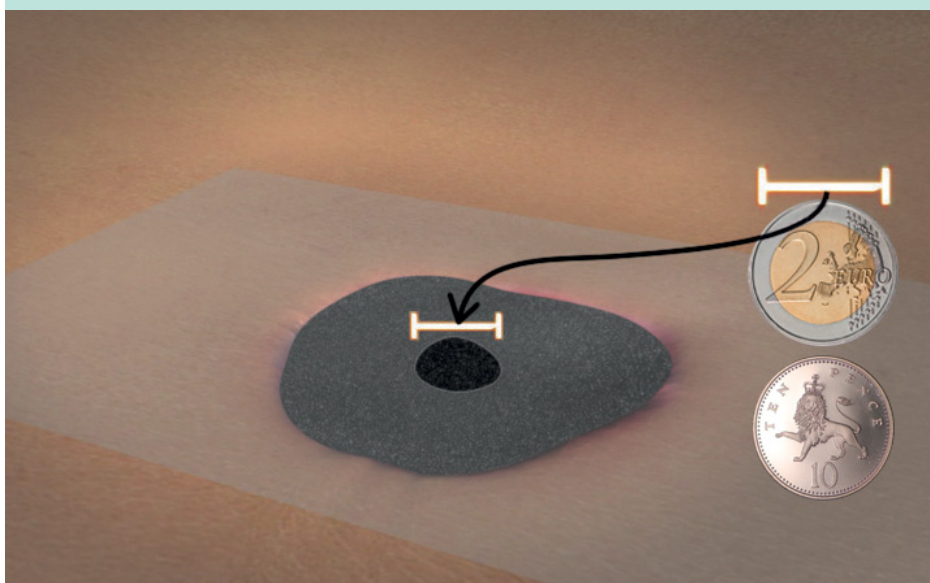


Consider the diameter of a 10 Pence or two Euro coin for size reference.

# Basic application guide for 3M™ V.A.C.® Dressing

**Step 7:** Remove backing Layer 1 and Layer 2 from 3M™ SensaT.R.A.C.™ Pad

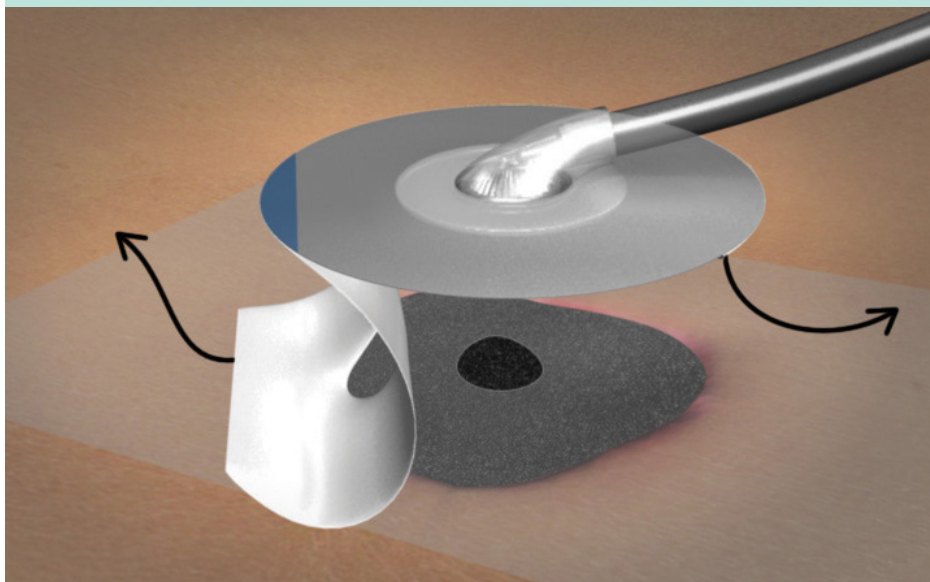


**Step 8:** Place 3M™ SensaT.R.A.C.™ Pad opening directly over hole

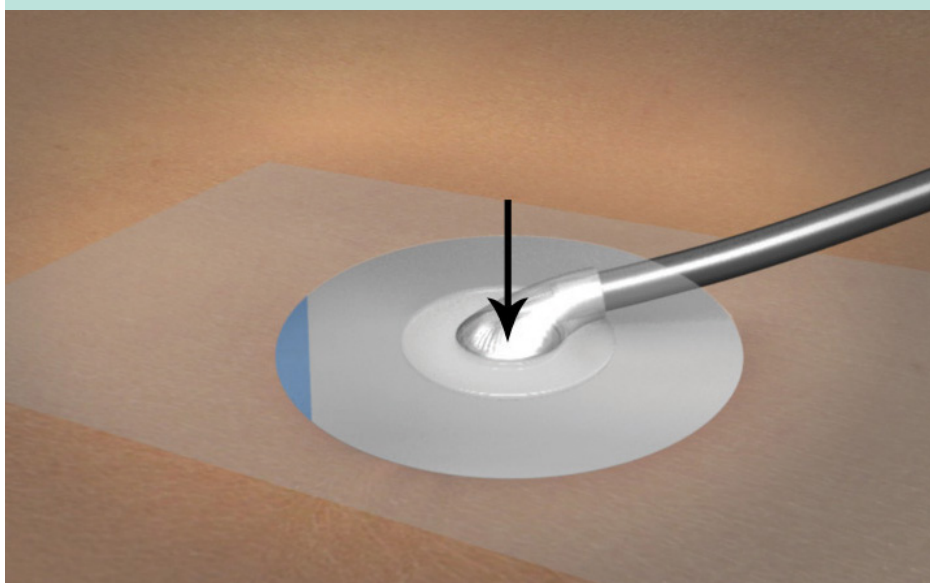


Pull blue tab to remove stabilisation layer. Record date and number of foam pieces used on the foam quantity label and place in the patient record.

# Basic application guide for 3M™ V.A.C.® Dressing

**Step 9:** Connect 3M™ SensaT.R.A.C™ Pads tubing to canister tubing

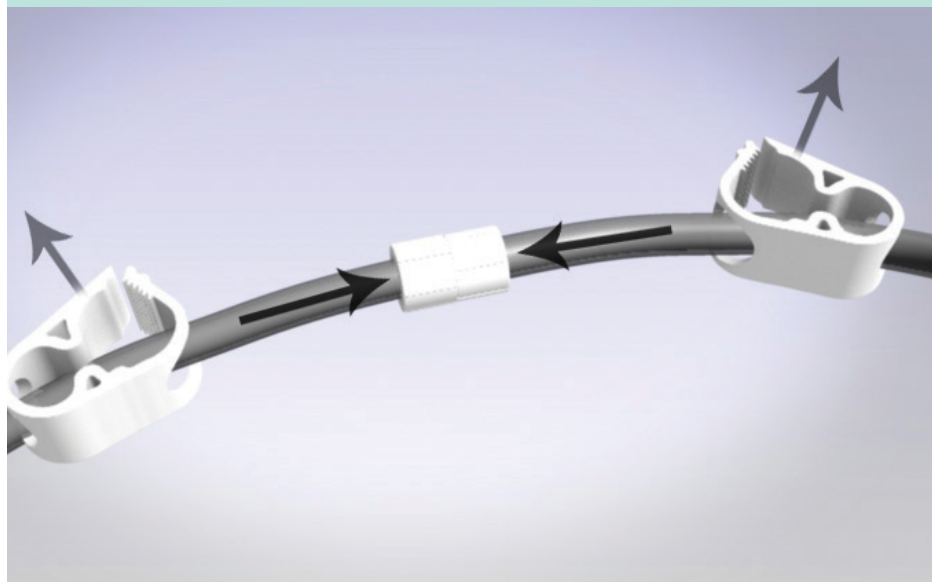

**Step 10:** Turn on power to 3M™ V.A.C.® Therapy Unit

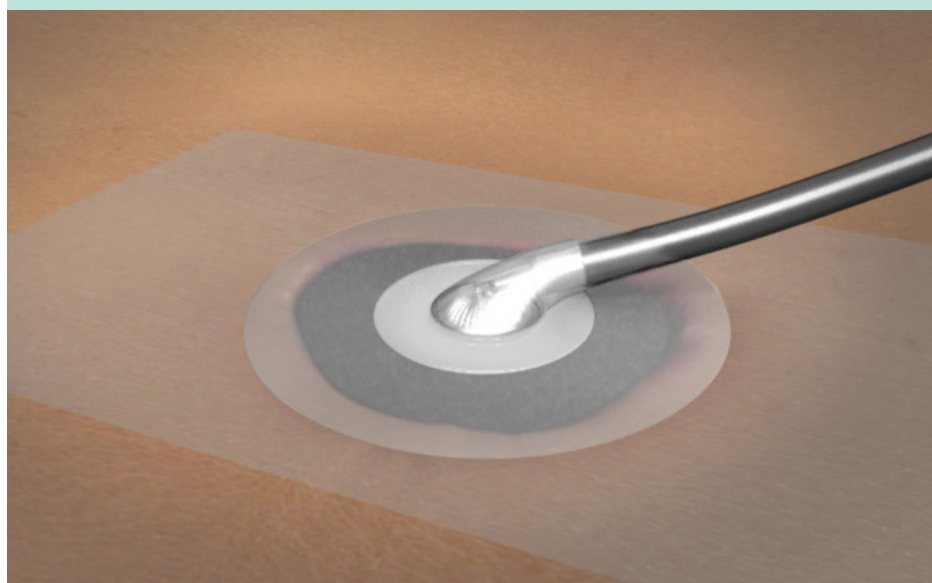

Set to the physician prescribed therapy settings to initiate therapy.



**Instructions for use and safety information:** This is a quick reference showing the basic steps in a typical dressing application. As with any device, it is important to read and understand the detailed instructions for use and safety information (including information on exposed vessels, organs, tendons, nerves and infection) applicable to your 3M™ V.A.C.® Therapy Unit and dressing application that can be found with the therapy unit, disposables carton or in the 3M™ V.A.C.® Therapy Clinical Guidelines.

**Questions and information:** For additional, detailed information, please refer to specific dressing instructions for use (contained in disposables cartons) and the 3M™ V.A.C.® Therapy Clinical Guidelines. These sources provide information concerning the proper application of all 3M™ V.A.C.® Dressing including advanced applications and techniques. If you have product questions, please contact your local representative.

**Note:** Specific indications, contraindications, warnings, precautions and safety information exist for these products and therapies. Please consult a clinician and product instructions for use prior to application. This material is intended for healthcare professionals.



For additional information, please contact your local Solventum representative.

**Solventum Medical Surgical**

Solventum UK Holding Company Limited  
Building 47, Charnwood Campus, 10 Bakewell Road  
Loughborough, LE11 5RB, United Kingdom  
[solventum.com/en-gb](https://solventum.com/en-gb)

Solventum Ireland Limited  
2 Cumberland Place, Fenian Street, Dublin 2, D02, H0V5, Ireland  
[solventum.com/en-ie](https://solventum.com/en-ie)

© Solventum 2026. Solventum and the S logo are trademarks of Solventum or its affiliates. 3M and the 3M logo are trademarks of 3M. All other trademarks are owned by their respective owners.  
PRA-PM-EU-00588 (12/21)