

Solventum™ V.A.C.® Ula™ Therapy Unit using Solventum™ Veraflo Cleanse Choice™ Dressing Kit

Disclaimer: This is intended for use as a supplement. Please refer to the instructions for use and user manual for full safety information.

1. Prepare the wound and apply 3M™ Cavilon™ No Sting Barrier Film

- Perform thorough wound and periwound area cleaning per physician order or institution protocol prior to each dressing application.
- To optimize therapy, consider removing or cross hatching eschar (make several shallow slits) per local guidelines or as directed by treating clinician. Skin should be clean and dry prior to application of Cavilon No Sting Barrier Film. Wipe uniform coating of film over the entire area of concern.
- Allow barrier to thoroughly dry before applying drape.*



2. Place the contact layer

- All exposed superficial vessels and organs in or around the wound must be completely covered and protected prior to administration of Veraflo Therapy.
- Assess wound dimensions and pathology, including the presence of undermining or tunnels. Do not place any foam dressing into blind/unexplored tunnels.
- Cut the cover layer (thick or thin) to fill explored tunnels and undermining. Be sure foam in tunnels contacts cover layer.
- Cut wound contact layer as needed to cover the entire wound bed and gently place in wound.



3. Select a Veraflo Cleanse Choice Dressing Kit size appropriate for the wound

- Select the Veraflo Cleanse Choice Dressing Kit (medium or large) based on the depth of the wound.
- Place the foam on top of the contact layer to determine the correct dressing size.
- Size the dressing layers as needed to allow gentle placement into the wound without firm packing of the foam or overlapping onto intact skin.



*Cavilon No Sting Barrier Film is extremely flammable. Allow product to dry for minimum 90 seconds before using any device that could provide a source.

**If adjunct non-adherent material used under contact layer, must be compatible with solution to be instilled and meshed, porous or fenestrated to allow for fluid removal.

Note:
Dressings should be changed every 48–72 hours, but no less than three times per week with frequency adjusted by the clinician as appropriate.
Do not use the wound contact layer with holes in tunnels or undermined area.
Do not use Octenisept, hydrogen peroxide or solutions that are alcohol-based or contain alcohol.
Ensure irrigation fluid or treatment solutions are fully removed from the dressing prior to stopping negative pressure wound therapy.

3. Select a Veraflo Cleanse Choice Dressing Kit size appropriate for the wound (continued)

- Do not cut or tear the foam over the wound. Away from wound site, rub foam edges to remove any fragments or loose particles that may fall into or be left in the wound upon dressing removal.
- Do not overpack the wound cavity.
- If using multiple pieces of foam, ensure foam-to-foam contact between adjacent pieces of foam for even distribution of fluid and negative pressure.
- Always note the total number of pieces of foam used in the wound and document on Foam Quantity Label and patient's chart.



4. Apply the Solventum™ Veraflo™ Drape

- Trim the drape to cover the foam and at least an additional 3–5 cm border of intact periwound tissue.
- Remove layer 1 to expose the adhesive and apply to cover layer and intact skin while ensuring skin wrinkles and folds are smoothed.
- Remove layer 2 and pat the drape to ensure an occlusive seal.
- Remove the ruler/handling bars.
- Smooth any wrinkles or creases in the dressing to prevent leaks. If leak source is identified, patch with additional drape to assure seal integrity.



5. Place the Solventum™ VeraT.R.A.C.™ Pad

- Choose pad application site. Give particular consideration to tubing positioning to allow for optimal flow and avoid placement over bony prominences or within creases in the tissue.
- Cut an approximately 2.5 cm hole through the drape to affix VeraT.R.A.C. pad. Cut a hole rather than a slit. A slit may self-seal during therapy.
- Remove both backing layers 1 and 2 to expose adhesive.
- Position and apply VeraT.R.A.C. pad over the 2.5 cm hole.
- Pull back on blue tab to remove pad stabilization layer.



What you'll need to initiate therapy



Solventum™ V.A.C.® Ultra™ Therapy Unit



Solventum™ V.A.C.® Canister with Gel, 500mL or 1000mL



Solventum™ Instillation Cassette



Topical wound solution



Solventum™ Veraflo Cleanse Choice™ Dressing Kit

Setting up the therapy unit

1. Press and hold the power button until light comes on and wait until a Start Up or New Patient Screen is shown



2. Once the Start Up or New Patient screen is visible, attach the Installation Cassette



3. Extend the solution container hanger arm



4. Spike and hang the solution container bag/bottle



5. Connect the installation line and then the V.A.C.® Therapy line. Align, push, and twist



6. Attach the 500 mL or 1000 mL canister
Select the prescribed therapy settings and initiate therapy. Refer to the V.A.C.® Ultra Therapy Unit User Manual for detailed instructions.

Note: Refer to the Instructions for Use for complete details regarding Dressing Application.



Risks may include allergic reactions and bleeding complications. See complete indications, limitations and important safety info: eifu.solventum.com/index.aspx?docid=5034 ▶




Initiating Solventum™ Smart Instill™ Feature

Start Up



Select Veraflo Therapy.

Therapy Setup

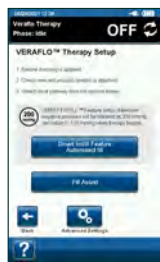

Select Smart Instill Feature.

Smart Instill Feature




Step 1
Select canister size and confirm canister is unused.



Step 2
Smart Instill Feature will initiate Solventum™ Seal Check™ Feature.



Step 3
The Smart Instill Feature screen will estimate and display the recommended instillation volume. If the user does not select Start, the Smart Instill Feature will automatically instill displayed instillation volume after 10 seconds.

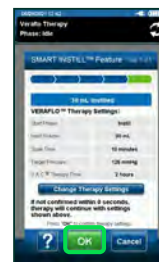

Step 4
Select OK to accept the suggested settings. Veraflo Therapy will begin automatically with settings displayed if user does not select OK after 10 minutes.

Initiating Veraflo Therapy

Veraflo Therapy – advanced settings overview (optional)



Select advanced settings.



Adjust pressure, intensity and therapy mode. Select OK to accept settings.



Turn fill assist off, select start phase. Adjust instill volume, soak time and V.A.C.® Therapy time.

Information screens – Veraflo Therapy



Select i button from the home screen to continue to the therapy summary tab.



Use the therapy summary tab to review a cumulative summary of therapy provided.



Use current settings tab to review the current therapy settings. Select change settings to continue to the confirm settings screen.