

Dressing selection for Solventum™ Veraflo™ Therapy

Understand the differences between the three dressing categories that support negative pressure wound therapy (NPWT) with instillation and dwell.

	Solventum™ Veraflo Cleanse Choice™ Dressing Kit 	Solventum™ Veraflo Cleanse™ Dressing Kit 	Solventum™ Veraflo™ Dressing Kit 
Purpose	<ul style="list-style-type: none"> Hydromechanically removes debris and promotes granulation tissue growth 	<ul style="list-style-type: none"> General wound cleansing for complex geometries 	<ul style="list-style-type: none"> General wound cleansing
Wound treatment goal	<ul style="list-style-type: none"> Cleansing by facilitating removal of wound exudate and infectious material Granulation tissue formation Extract non-viable tissue 	<ul style="list-style-type: none"> Cleansing and granulation tissue formation Fill explored tunnels & undermining with a rod shaped foam with perforations for easy separation 	<ul style="list-style-type: none"> Cleansing and granulation tissue formation
Dressing indication statement	<ul style="list-style-type: none"> Provides hydromechanical removal of infectious materials, non-viable tissue and wound debris Reduces the number of surgical debridements required, while promoting granulation tissue formation, creating an environment that promotes wound healing 	<ul style="list-style-type: none"> Provides vacuum assisted drainage and controlled delivery of topical wound treatment solutions and suspensions over the wound bed 	<ul style="list-style-type: none"> Provides vacuum assisted drainage and controlled delivery of topical wound treatment solutions and suspensions over the wound bed
Debridement required when eschar/necrotic tissue present before applying dressing?	No, but crosshatch eschar before applying therapy for optimal results	Yes	Yes
Reduce the need for surgical debridement cleared by FDA?	Yes	No	No
Provider CPT® Code	<ul style="list-style-type: none"> 97605 size ≤ 50 sq cm 97606 size > 50 sq cm 	<ul style="list-style-type: none"> 97605 size ≤ 50 sq cm 97606 size > 50 sq cm 	<ul style="list-style-type: none"> 97605 size ≤ 50 sq cm 97606 size > 50 sq cm
Sizes and SKU	<ul style="list-style-type: none"> Medium: ULTVCC05MD Large: ULTVCC05LG 	<ul style="list-style-type: none"> Medium: ULTVCL05MD 	<ul style="list-style-type: none"> Small: ULTVFL05SM Medium: ULTVFL05MD Large: ULTVFL05LG

Refer to each dressing's instructions for use. For more information, visit eifu.solventum.com.

	Solventum™ Veraflo Cleanse Choice™ Dressing Kit 	Solventum™ Veraflo Cleanse™ Dressing Kit 	Solventum™ Veraflo™ Dressing Kit 
Foam composition and dressing components	<p>Medium: Blue compressed single piece block foam with a circular hole pattern contact layer</p> <ul style="list-style-type: none"> • 18.0 cm x 12.5 cm x 1.6 cm • 3 sheets Solventum™ Veraflo™ Drape • 1 Solventum™ VeraT.R.A.C.™ Pad • 4pk 3M™ Cavilon™ No Sting Barrier Film • 1 disposable ruler/foam quantity label <p>Large: Blue compressed block foam in three pre-slit layers: a circular hole pattern contact layer plus two cover layers (thin and thick)</p> <ul style="list-style-type: none"> • 25.6 cm x 15.0 cm x varying thicknesses (0.8 cm – 1.6 cm) • 6 sheets Veraflo Drape • 1 Solventum™ VeraT.R.A.C.™ Duo Pad • 5pk 3M Cavilon No Sting Barrier Film • 1 disposable ruler/foam quantity label 	<p>Medium: Gray compressed tubular coiled foam</p> <ul style="list-style-type: none"> • 61 x 3.2 cm diameter • 3 sheets Veraflo Drape • 1 VeraT.R.A.C. Pad • 2pk Cavilon No Sting Barrier Film • 1 disposable ruler/foam quantity label 	<p>Small: Black spiral-cut foam, not compressed</p> <ul style="list-style-type: none"> • 11.2 x 7.7 x 1.8 cm per piece (2 pieces spiral) • Each piece: 24 cm uncoiled; 18 cm long bridge • 2 sheets Veraflo Drape • 1 VeraT.R.A.C. Pad • 1pk Cavilon No Sting Barrier Film • 1 disposable ruler/foam quantity label <p>Medium: Black spiral-cut foam, not compressed</p> <ul style="list-style-type: none"> • 17.4 x 14.7 x 1.8 cm per piece (2 pieces spiral) • Each piece: 80 cm uncoiled; 74 cm long bridge • 3 sheets Veraflo Drape • 1 VeraT.R.A.C. Pad • 2pk Cavilon No Sting Barrier Film • 1 disposable ruler/foam quantity label <p>Large: Black block foam pre slit into two layers, not compressed</p> <ul style="list-style-type: none"> • 25.6 x 15.0 x 1.6 cm per piece (oval - 2 pieces) • 5 sheets Veraflo Drape • 1 VeraT.R.A.C. Duo Pad; 4pk Cavilon No Sting Barrier Film • 1 disposable ruler/foam quantity label
Compatible with automated topical solution instillation through Solventum™ Smart Instill™ Feature?	Yes	Yes	Yes
Fill assist start points	<ul style="list-style-type: none"> • Medium: 66 mL • Large: 150 mL (1.6 cm cover layer); 75 mL (0.8 cm cover layer); 42 mL (0.8 cm wound contact layer) 	<ul style="list-style-type: none"> • Medium: 65-130 mL (1/2 rod); 130-250 mL (full rod) 	<ul style="list-style-type: none"> • Small: 12-80 mL (1 piece); 26-160 mL (2 pieces) • Medium: 38-240 mL (1 piece); 80-450 mL (2 pieces) • Large: 55-350 mL (1 piece); 110-500 mL (2 pieces)

Learn more at [solventum.com/Veraflo](https://www.solventum.com/Veraflo)

Solventum Advanced Wound Care
12930 IH10 W
San Antonio, TX 78249
USA

Phone 1-800-275-4524
Web [Solventum.com](https://www.solventum.com)

Refer to each dressing's instructions for use. For more information, visit [eifu.solventum.com](https://www.eifu.solventum.com)

For any reimbursement questions relating to the Solventum Medical Surgical portfolio of products, please contact our **Reimbursement Hotline at 1-800-668-6812**.

Important: This content is for informational purposes only and represents no statement, promise or guarantee by Solventum concerning the levels of reimbursement, payment, calculations, eligibility, charge or that these policies and codes will be appropriate for specific services or products or that reimbursement will be made. Before filing any claim, providers should verify current requirements and policies with the payor.

Note: Specific indications, contraindications, warnings, precautions and safety information exist for these products and therapies. Please consult a clinician and product instructions for use prior to application. Rx only.

©Solventum 2026. Solventum, the S logo and V.A.C. are trademarks of Solventum or its affiliates. 3M is a trademark of 3M. Other trademarks are the property of their respective owners. US_EN_70-2013-1242-1