

With the 3M™ V.A.C.® UIta Therapy System

A world leader in
skin and wound care
right by your side



Science-
based
solutions

Our comprehensive
portfolio of advanced
wound care solutions is
at the forefront
of innovation.



Ongoing
support

Our team is available
to assist with clinical and
technical support.



World-class
education

Empowering you with
hands-on training and
award-winning education.

For more information,
visit [solventum.com](https://www.solventum.com)

Note: Specific indications, contraindications, warnings, precautions and safety information exist for these products and therapies. Please consult a clinician and product instructions for use prior to application. This material is intended for healthcare professionals.

© Solventum 2026. Solventum and the S logo are trademarks of Solventum or its affiliates. 3M and the 3M logo are trademarks of 3M. All other trademarks are owned by their respective owners.

Topical wound solutions for use with 3M™ Veraflo™ Therapy

Topical wound solutions

The following topical solutions are compatible with 3M™ Veraflo™ Therapy based on 3M in-house testing of the Veraflo Therapy disposables with respect to mechanical properties, biocompatibility and solution interaction.*

Generic solution class	Considerations for use with 3M™ Veraflo™ Therapy
Hypochlorite-based solutions* (e.g. hypochlorous acid, sodium hypochlorite)	<ul style="list-style-type: none"> Based on results of compatibility testing, Dakin's Solution should not be used in concentrations greater than 0.125% (quarter strength) Consider using the fewest irrigation cycles and minimising hold times to the lowest level that is clinically relevant
Silver nitrate (0.5%)	Silver nitrate is light sensitive. Protect solution container and 3M™ V.A.C. Veralink™ Cassette tubing from light during use of silver nitrate.
Biguanides (Polyhexanide)	Solution container can be: <ul style="list-style-type: none"> Spiked directly Connected using a spike adapter if available Transferred to a container having a spike port For compatibility assessment, polyhexanide solution was tested in final concentrations equal to or less than 0.1%.
Lidocaine HCl	<ul style="list-style-type: none"> For compatibility assessment, lidocaine HCl was tested in final concentrations equal to or less than 0.1% in saline Toxicity concerns may exist; consult with your pharmacist and/or prescribing physician for solution and patient-specific considerations when using lidocaine HCl as an additive to saline instillation solutions No device-related considerations for use with Veraflo Therapy
Cationic solutions (Octenidine)	For compatibility assessment, Octenilin® solution was tested in final concentrations equal to or less than 0.05% Octenidine.
Isotonic solutions	May need to be connected using a spike adapter.
Acetic acid	For compatibility assessment, acetic acid was tested in final concentrations equal to or less than 0.25% in sterile water.
Benzalkonium chloride (0.1%)	For compatibility assessment, benzalkonium chloride was tested in final concentrations equal to or less than 1:200,000 dilution.

*Hypochlorous acid solutions applied frequently at high concentrations can lead to significant material degradation. Consider utilizing concentrations and exposure durations as low as clinically relevant. Testing indicates that the concentration of hypochlorous acid, a preservative of the solution in the bottle, may be significantly reduced after prolonged contact with the fluid path components of the Veraflo Therapy delivery system. Due to the reactive nature of hypochlorous acid solutions, exposure to the fluid path components of the Veraflo Therapy delivery system may impact the final active or preservative concentration of the solution.

*Solventum is solely responsible for the solution compatibility information provided herein. Always refer to the solution manufacturer's prescribing information when deciding whether to administer any solution with the 3M™ V.A.C.® Ultra Therapy System and for appropriate safety, efficacy and dosing information. Listing of specific solutions is not an endorsement of a specific solution or an indication of clinical efficacy or safety.

With the 3M™ V.A.C.® UIta Therapy System

A world leader in
skin and wound care
right by your side



	Science-based solutions	Our comprehensive portfolio of advanced wound care solutions is at the forefront of innovation.
	Ongoing support	Our team is available to assist with clinical and technical support.
	World-class education	Empowering you with hands-on training and award-winning education.

Topical wound solutions for use with 3M™ Veraflo™ Therapy

For more information,
visit [solventum.com](https://www.solventum.com)

Note: Specific indications, contraindications, warnings, precautions and safety information exist for these products and therapies. Please consult a clinician and product instructions for use prior to application. This material is intended for healthcare professionals.

© Solventum 2026. Solventum and the S logo are trademarks of Solventum or its affiliates. 3M and the 3M logo are trademarks of 3M. All other trademarks are owned by their respective owners.

With the 3M™ V.A.C.® Ultra Therapy System

A world leader in
skin and wound care
right by your side



	Science-based solutions	Our comprehensive portfolio of advanced wound care solutions is at the forefront of innovation.
	Ongoing support	Our team is available to assist with clinical and technical support.
	World-class education	Empowering you with hands-on training and award-winning education.

For more information,
visit [solventum.com](https://www.solventum.com)

Note: Specific indications, contraindications, warnings, precautions and safety information exist for these products and therapies. Please consult a clinician and product instructions for use prior to application. This material is intended for healthcare professionals.

© Solventum 2026. Solventum and the S logo are trademarks of Solventum or its affiliates. 3M and the 3M logo are trademarks of 3M. All other trademarks are owned by their respective owners.

Topical wound solutions for use with 3M™ Veraflo™ Therapy